[www.drivemedical.com](http://www.drivemedical.com)

drive  
MEDICAL DESIGN & MANUFACTURING

facial mask

máscara facial

masque facial

EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

[www.drivemedical.com](http://www.drivemedical.com)

drive  
MEDICAL DESIGN & MANUFACTURING

## INTENDED USE

The Full Face CPAP Mask is intended to be used for adult patients who have been prescribed CPAP or bi-level therapy by a physician or respiratory therapist. It is used in conjunction with continuous positive airway pressure (CPAP) or bi-level system in home, hospital and institutional environments.

## CAUTION

US federal law restricts this device to sale by or on the order of a physician.

## CONTRAINDICATIONS

This mask may not be suitable for patients with the following conditions: excessive reflux, impaired cough reflex and hiatal hernia. It should not be used if the patient is unresponsive, obtunded, unable to remove the mask or uncooperative.

## WARNINGS

- Contact a health care professional if chest discomfort, shortness of breath, stomach distention, belching, or severe headache are experienced.

- Avoid eating or drinking three hours before using this mask to minimize the risk of vomiting during sleep.

- Do not use this mask if you are taking drugs that may cause vomiting.

- Do not use this mask unless the CPAP or Bi-level system is turned on and operating properly. When the system is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.

## CLEANING INSTRUCTIONS

Before use, clean the mask following the instructions below.

- Detach headgear from mask.


- Handwash gently with mild soap and warm water, rinse the mask and components thoroughly.

- Leave to air dry before reusing.


- Do not use bleach, chlorine, alcohol or aromatic-based solutions, these solutions may cause hardening and reduce the life of the product.

To remove the mask and headgear, simply unbuckle both buckles from the mask frame and pull the upper head strap away from the head.


To reuse the mask and headgear, simply place upper headstrap over head and hook up to both buckles.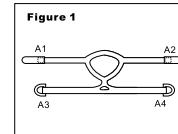


Place headgear with Velcro side down on a flat surface, detach 2 Velcro tapes on A1 & A2.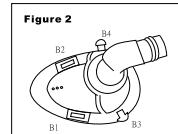


Identify headgear fastening positions on the mask.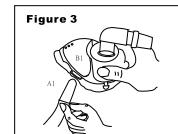


Pull the upper headgear strap A1 through the slot B1 on the mas, making sure the Velcro faces outward.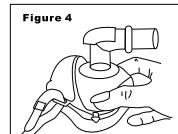


Fasten the Velcro tape.

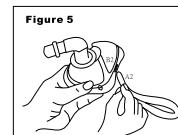
Pull the upper headgear strap through the slot B2 on the mask, making sure the Velcro faces outward.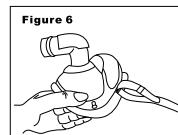


Fasten the Velcro tape.

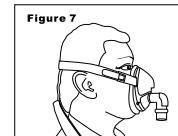
Place the headgear over your head and adjust both upper headgear straps to a comfortable tension.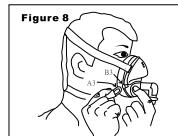


Hook the plastic buckle A3 onto B3 on mask frame.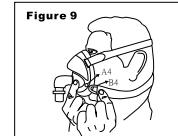


Hook the plastic buckle A4 onto B4 on mask frame.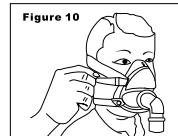

Adjust both again upper headgear straps A1 & A2 to a comfortable tension.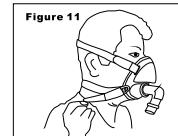

Adjust both lower headgear straps A3 & A4 to a comfortable tension.

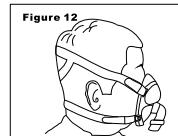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## 90 day limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for 90 days of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. The 90 Day Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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©2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.