

## QUICKIE GP/GPV

### A. FRONT SEAT HEIGHT (For standard frames only)

| Caster Fork Mounting Hole | 3" Micros 5 1/4" Fork | 3" Micros 6" Fork | 3" Micros 7" Fork | 5" LP Poly 5 1/4" Fork | 5" LP Poly 6" Fork | 5" LP Poly 7" Fork | 6" Pneu 5 1/4" Fork | 6" Pneu 6" Fork | 6" Pneu 7" Fork | 6" Poly 5 1/4" Fork | 6" Poly 6" Fork | 6" Poly 7" Fork |
|---------------------------|-----------------------|-------------------|-------------------|------------------------|--------------------|--------------------|---------------------|-----------------|-----------------|---------------------|-----------------|-----------------|
| Top                       | 17 1/16"              | 17 7/8"           | 18 7/8"           | 17 3/16"               | 18"                | 19"                | 17 3/4"             | 18 9/16"        | 19 9/16"        | 17 5/8"             | 18 7/16"        | 19 7/16"        |
| Bottom                    | 18 1/16"              | 18 7/8"           | 19 7/8"           | 18 3/16"               | 19"                | 20"                | 18 3/4"             | 19 9/16"        | 20 9/16"        | 18 5/8"             | 19 7/16"        | 20 7/16"        |


### B. REAR SEAT HEIGHT (For standard frames and axle position only)

| Rear Axle Hole Positioning | 22" Pneu | 24" Latex Tubular | 24" HP Clincher | 24" Pneu | 24" HT Tennis/B-ball Turbo Tread | 26" Latex Tubular | 26" HP Clincher | 26" Pneu | 26" HT Tennis/B-ball Turbo Tread |
|----------------------------|----------|-------------------|-----------------|----------|----------------------------------|-------------------|-----------------|----------|----------------------------------|
| Top                        | 15 3/4"  | 15 7/8"           | 16"             | 16 1/8"  | 16 1/4"                          | 16 7/8"           | 16 7/8"         | 17 1/4"  | 17 5/8"                          |
| Middle                     | 16"      | 16"               | 16 5/8"         | 16 3/4"  | 16 7/8"                          | 17 1/2"           | 17 1/2"         | 17 7/8"  | 18 1/4"                          |
| Bottom                     | 16 5/8"  | 17 1/8"           | 17 1/4"         | 17 3/8"  | 17 1/2"                          | 18 1/8"           | 18 1/8"         | 18 1/2"  | 18 7/8"                          |

### C. OVERALL WIDTH

| Frame Width                        | 12"     | 13"     | 14"     | 15"     | 16"     | 17"     | 18"     | 19"     | 20"     |
|------------------------------------|---------|---------|---------|---------|---------|---------|---------|---------|---------|
| Mag wheel with aluminum handrims   | 19 1/2" | 20 1/2" | 21 1/2" | 22 1/2" | 23 1/2" | 24 1/2" | 25 1/2" | 26 1/2" | 27 1/2" |
| Spoke wheel with aluminum handrims | 20 1/2" | 21 1/2" | 22 1/2" | 23 1/2" | 24 1/2" | 25 1/2" | 26 1/2" | 27 1/2" | 28 1/2" |

Actual measurements may vary up to 3/8". Please consult your order form or customer service for combinations and restrictions. Specifications subject to change without notice.

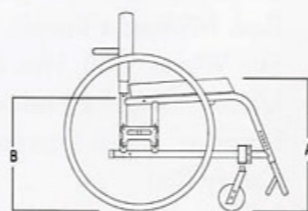

## QUICKIE GP SWING-AWAY

### A. FRONT SEAT HEIGHT (For standard frames only)

| Caster Fork Mounting Hole | 5" LP Poly 5 1/4" Fork | 5" LP Poly 6" Fork | 5" LP Poly 7" Fork | 6" Pneu 5 1/4" Fork | 6" Pneu 6" Fork | 6" Pneu 7" Fork | 6" Poly 5 1/4" Fork | 6" Poly 6" Fork | 6" Poly 7" Fork | 8" Pneu 5 1/4" Fork | 8" Pneu 6" Fork | 8" Pneu 7" Fork |
|---------------------------|------------------------|--------------------|--------------------|---------------------|-----------------|-----------------|---------------------|-----------------|-----------------|---------------------|-----------------|-----------------|
| Top                       | 15 9/16"               | 16 3/8"            | 17 3/8"            | 16 1/8"             | 16 15/16"       | 17 15/16"       | 16"                 | 16 13/16"       | 17 13/16"       | 17 1/8"             | 17 15/16"       | 18 15/16"       |
| Bottom                    | 16 9/16"               | 17 3/8"            | 18 3/8"            | 17 1/8"             | 17 15/16"       | 18 15/16"       | 17"                 | 17 13/16"       | 18 13/16"       | 18 1/8"             | 18 15/16"       | 19 15/16"       |


### B. REAR SEAT HEIGHT (For standard frames and axle position only)

| Rear Axle Hole Positioning | 22" Pneu | 24" Latex Tubular | 24" HP Clincher | 24" Pneu | 24" HT Tennis/B-ball Turbo Tread | 26" Latex Tubular | 26" HP Clincher | 26" Pneu | 26" HT Tennis/B-ball Turbo Tread |
|----------------------------|----------|-------------------|-----------------|----------|----------------------------------|-------------------|-----------------|----------|----------------------------------|
| Top                        | 15 7/8"  | 16 3/8"           | 16 1/2"         | 16 5/8"  | 16 3/4"                          | 17 3/8"           | 17 3/8"         | 17 3/4"  | 18 1/8"                          |
| Middle                     | 16 1/2"  | 17"               | 17 1/8"         | 17 1/4"  | 17 3/4"                          | 18"               | 18"             | 18 3/8"  | 18 3/4"                          |
| Bottom                     | 17 1/8"  | 17 5/8"           | 17 3/4"         | 17 7/8"  | 18"                              | 18 5/8"           | 18 5/8"         | 19"      | 19 3/8"                          |

### C. OVERALL WIDTH

| Frame Width                        | 12"     | 13"     | 14"     | 15"     | 16"     | 17"     | 18"     | 19"     | 20"     |
|------------------------------------|---------|---------|---------|---------|---------|---------|---------|---------|---------|
| Mag wheel with aluminum handrims   | 19 1/2" | 20 1/2" | 21 1/2" | 22 1/2" | 23 1/2" | 24 1/2" | 25 1/2" | 26 1/2" | 27 1/2" |
| Spoke wheel with aluminum handrims | 20 1/2" | 21 1/2" | 22 1/2" | 23 1/2" | 24 1/2" | 25 1/2" | 26 1/2" | 27 1/2" | 28 1/2" |

Actual measurements may vary up to 3/8". Front seat height must be equal to or greater than rear seat height. Front seat height cannot be more than 2" higher than rear seat height. Please consult your order form or customer service for combinations and restrictions. Specifications subject to change without notice.

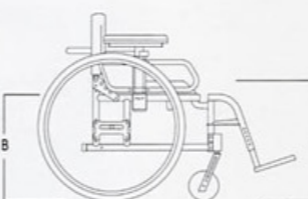

Sunrise Medical Limited  
High Street Wollaston  
West Midlands DY8 4PS  
England  
Tel +44 (0) 1384 446688  
Fax +44 (0) 1384 446699  
www.sunrisemedical.co.uk



# BECAUSE LIFE IS A PARTICIPATION SPORT.



QUICKIE GPV COMPETITION

Think of the Quickie GPV as the sports chair that's built for the toughest game of all: your everyday life.

The Quickie GPV's frame is lightweight but rigid for rock-solid handling. You can adjust the center of gravity, axle position and chair angle to suit your preferences. And the GPV wouldn't be a Quickie if you couldn't customize it with a variety of footrests, casters, front frame bends and special sizes. You can even opt for the Quickie GPV Competition, the ultimate performance chair to fire up the competitor in you.

Now you can Get Out There and show them what you've got, on and off the court. You and your Quickie GPV.

**QUICKIE**  
The RealChair People.





## QUICKIE



## GPV

- Weight: 21 1/2 lbs. (with steel rear axles), 19 3/4 lbs. (with titanium rear axles)
- Color Options: Blue, Black, Red, Midnight Purple, Silver, Sky White, Teal, Hot Pink, Ultra Yellow, Lavender, Blue Sapphire, Black Diamond, Candy Red, Black Splatter, Red Splatter, Sky White Splatter
- Lifetime frame warranty

For the ultimate performance chair try the Quickie GPV Competition.

Competition Package Includes:

- Squeeze frame
- 80° front frame bend
- Short frame length
- Quickie performance wheels

## QUICKIE GP

- Weight: 24 1/2 lbs.
- Color Options: Blue, Black, Red, Midnight Purple, Silver, Sky White, Teal, Hot Pink, Ultra Yellow, Lavender, Blue Sapphire, Black Diamond, Candy Red
- Lifetime frame warranty

## SWING AWAY



### Caster Options

Quickie offers a wide selection of casters in all different shapes and sizes. Try the 3" Micros for on-court maneuverability or opt for polyurethane or pneumatic casters on either aluminum or composite rims.



### Custom Frames

Each Quickie GPV frame is custom built for the user. Whether it's a tall, short, long, wide or squeezed frame you're after, Quickie can accommodate your needs. If you don't see it listed on the order form, just ask!



### Rigid Platform Footplate

Quickie's rigid platform footplate is designed to accommodate Quickie GP and GPV chairs with front frame bends greater than 75°. The footplate is angle adjustable to provide proper foot and leg positioning.



### Height Adjustable Armrest

Designed for easy adjustment, this detachable padded armrest can be moved up and down with ease. Available with either a 10" standard pad or a 14" full length pad.



### Folding Back

With the folding back option, portability is a snap. Just pull the wheels off, fold, and pop in the car. Folding backrest is available with or without push handles.



### Hanger Options

Quickie GP Swing-away is available with a variety of hanger options. Choose from 60°, 70° and 90° standard hangers, or try the 70° "V" style hanger.