

# Quick Use Guide


## Charging Method

### 1 Two Charging Methods

**Charging On-Chair:**  
When charging on-chair, plug the powercord into the wall socket and the charging cord into the bottom of the joystick.



**Charging Off-Chair**  
When charging off-chair, plug the power cord into the wall socket and the charging cord into the battery box.

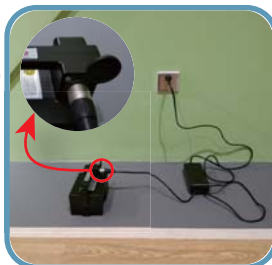

### Caution:

- Input Voltage : AC100~240V,50/60Hz.
- When the light turn green,it means that the battery is full.
- Do not use the wheelchair when it is charging.
- If the wheelchair is not used for a long period of time,turn off the power. Charge once a month at least

## Seat-belt Use Method

Always use your seat while driving !



To undo, press sides of buckle and slide female end out.



Insert male end of seatbelt into female end until it clicks into place.



**If you forget to unfasten your seat belt and get out of a wheelchair, you may fall. Please be careful to avoid !**

## Matters Need Attention

- Check wheelchair whether it is loose, sloshing, twisting before use.
- Check battery interface before use.
- Check the electricity is adequate before use.
- For initial users, suggest driving at low speed to avoid accidents.
- Turn off the power when going up and down the wheelchair .
- Don't make a sharp turn. Don't move sideways to avoid overturning.
- Do not start or stop electric wheelchairs frequently.
- The operation controller action is gentle, must not exert too hard, lest damage the controller.

JINMED®

# Quick Use Guide



※ All photos here are for reference only.  
Specifications are subject to the physical product.

CHANGZHOU ZHONGJIN MEDICAL CO.,LTD


Thanks for choosing our product.In order to use the wheelchair appropriately,please read this guidance carefully before using.

# Quick Use Guide

## Open And Use Of Wheelchairs





1 Take out the wheelchair from the package.

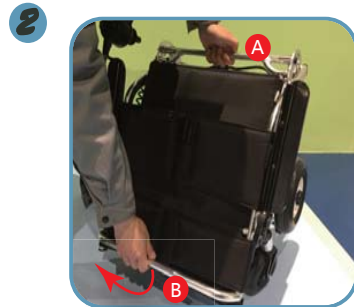


2 One hand holds the aluminum tube A, the other holds the aluminium tube B, and opens it outwards.



3 Make sure the frame is open and in place, and try to push the wheelchair and check it shake or fall off.

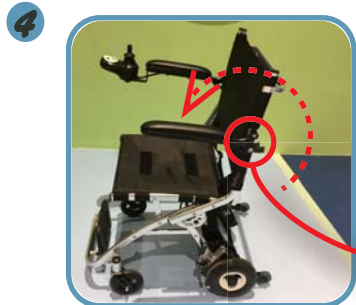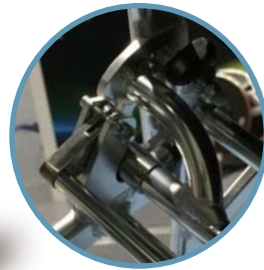

4 Grasp arms at their ends and rotate up and forward until they contact the stopper on the back post

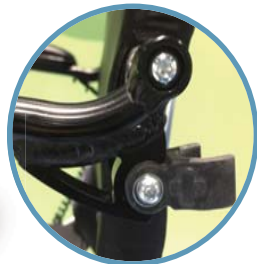


 **The frame must be opened in place, otherwise the wheelchair would be damaged and people would be injured.**

## Open And Use Of Wheelchairs



5 Flip the pedals to the horizontal level and install the outer cushion

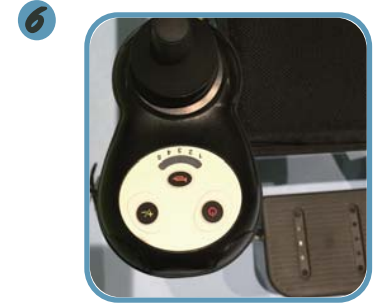

6 After sitting down, press the power supply key, operate the controller and drive the wheelchair.



7 By pushing the joystick, the user operate the wheelchair and arbitrarily in any direction



8 Controller handles can be swapped left and right according to user usage

## Manual /electric switching mode



Pull down to change to manual mode; pull up to change to electric mode

## Controller System Diagnosis

When the controller LED lights flicker, the abnormal situation may occur in the electric wheelchair. The nature of the abnormal condition can be classified according to the diagnosis number, and the corresponding problems and solutions can be found according to the table below. Electric wheelchairs are not allowed to drive until the problem is resolved.

Light signal description	Possible Causes	Remarks
The LED are all off. No sound	The power is off or in standby or the system is in sleep mode. The power line is in poor contact and the fuse has blown or burnt out.	
LED lights are all bright	The power is on. the product can be used normally.	The few lights are on ,means less battery.
Leftmost red light is the only light sound alarm	Extremely low battery	Charge the chair as soon as possible
Buzzer sounds 2 short rings light falshes twice	Left motor failure	Poor contact with left motor or disconnection of the wires
Buzzer sounds 3 short rings light falshes three times	Left brake failure	Poor contact with left brake, or brake power failure or switch is damaged
Buzzer sounds 4 short rings light falshes four times	Right motor failure	Poor contact with right motor or disconnection of the wires
Buzzer sounds 5 short rings light falshes five times	Right brake failure	Poor contact with right brake, or brake power failure or switch is damaged
Buzzer sounds 6 short rings light falshes six times	Controller overflow-protected state	Check the brakes, whether the motor drive is stuck, maybe a controller problem.
Buzzer sounds 7 short rings light falshes seven times	Joystick gimbal failure	The joystick gimbal is not reset or broken, and maybe the connector is loose.
Buzzer sounds 8 short rings light falshes eight times	Controller	Please contact your distributor.
Buzzer sounds 9 short rings light falshes nine times	Left motor Hall line failure	The controller did not detect a valid signal
Buzzer sounds 10 short rings light falshes ten times	Right motor Hall line failure	The controller did not detect a valid signal

## Controller Operating Guide




※ All photos here are for reference only.  
※ Specifications are subject to the physical product.

CHANGZHOU ZHONGJIN MEDICAL CO.,LTD

Thanks for choosing our product. In order to use the wheelchair appropriately, please read this guidance carefully before using.

## Controller Operation Instruction

### 1 Operation interface

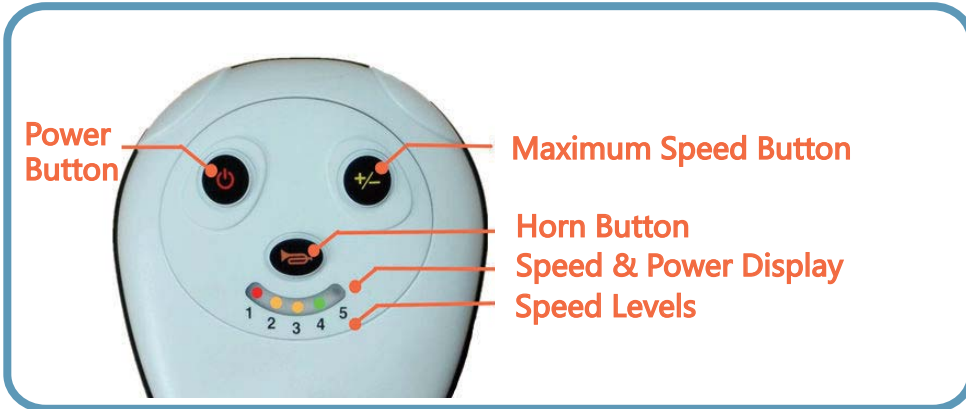

**!** Carefully clean the joystick with a damp cloth & mild detergent. No corrosive or alcohol type cleansers.


### 2 Power Switch Function:

- 1) Press the power button to turn the chair on. Indicator lights will light and horn will sound.
- 2) Press the power button again to turn off the chair. Indicator lights will go off and chair functions will cease.

**!** Turn on the power when the joystick is not in the middle, an alarm will occur. Release the joystick and bring it back to the middle, the error will disappear.

**Dormant mode:** If the joystick does not have any action, after 20 minutes, the controller will automatically turn off the power, enter the dormant mode, press the power switch button, you can wake up the control system from dormant mode.

### 3 Use of loudspeakers


1 Press the horn button  and the horn will continue to ring until release the horn button.






## Controller Operation Instruction

### 4 Driving speed adjustment

According to the user's habits, the user can adjust the maximum speed of the electric wheelchair. The current selected maximum speed will

be displayed in the speed limit indicator light . The user

can adjust the speed by using the add or decrease speed button . Adjustment mode is circular, each press a button, gear plus one, from 1 to 5 cycles (1 is the slowest and 5 is the fastest).

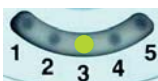Speed 1		Speed 4	
Speed 2		Speed 5	
Speed 3			

### 5 Battery Indicator:

The same lights as the speed indicator are used for the battery level indicator. When the lights are coloured they show the charge level of the battery. When all 5 lights are lit the chair is fully charged. The lights go out as the battery charge diminishes. When the red light is the only light lit, it is important to charge the chair as soon as possible. To ensure that the wheelchair can be used normally.

**!** If you need to drive long distances, be sure to charge first !

**!** Do not charge after you run out of electricity !