


FlexTable[®]


A TABLE FOR ALL

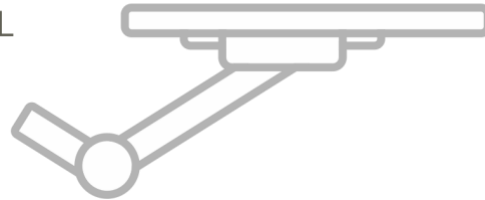

Specifications




Assembled Weight: 97lbs

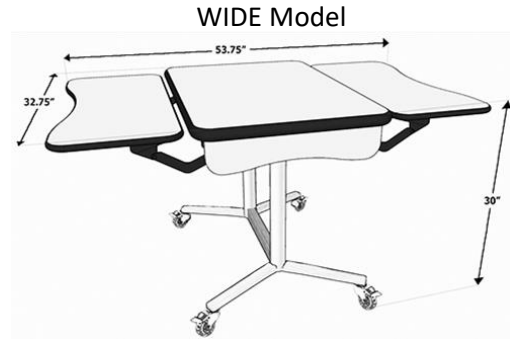

Assembled Weight: 102.5lbs

Features

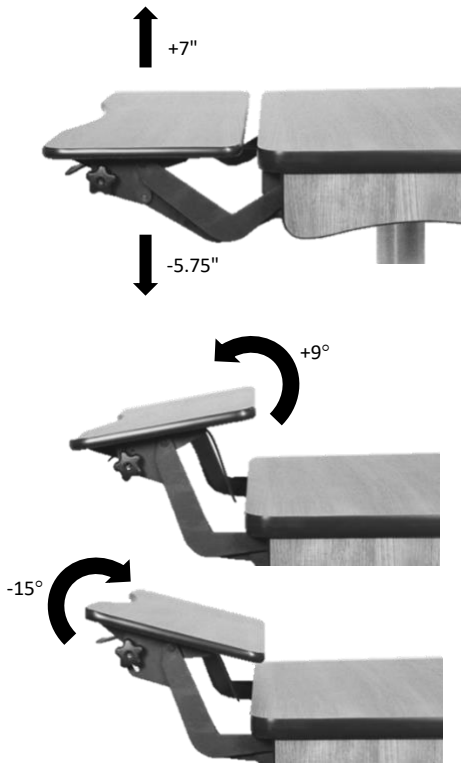

Height Adjustable

Each flex tray is height adjustable with a travel range of 12.75" - 7" up and 5.75" down.

Tiltable

Each flex tray offers +9°/-15° of tilt for custom positioning.


Joinable


Any FlexTable® model can be easily joined together in less than 10 seconds.

Safety

To ensure safety to individuals and property, you must thoroughly read all sections of the user manual prior to assembly and use.

Hazard Symbols








 A **WARNING** symbol indicates the potential for property damage or personal injury.

 A **CAUTION** symbol indicates potential damage to hardware and tells you how to avoid the problem.

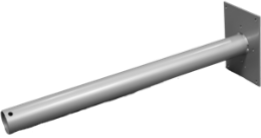WARNING

- **DO NOT** exceed the maximum listed weight capacity.
- **NEVER** sit or lean on *any* FlexTable® surface.
- **DO NOT** attempt to move table while caster locks are engaged.

Components

A	B	C	D
			
E	F	G	
			


Assembly

Remove components box and attached metal legs by cutting zip ties. Unpackage all components and make sure you have all supplies needed to assemble the FlexTable®. This includes:

- A. 2 - metal legs
- B. 16 - Phillips head screws
- C. 1 - Phillips head screwdriver
- D. 1 - spacer
- E. 6 - bracket/ bolt/ nut combos
- F. 2 - 9/16 wrenches
- G. 4 - feet w/ casters


*Scissors are required to complete assembly.

1. Place one leg (A) plate side down onto the bottom of the FlexTable® top, lining up the holes on the leg plate with the holes on the FlexTable® top as shown in Diagram (1). Ensure the hole on the inside of the leg is facing inward. Using 8 screws (B), secure the leg to the base of the table using the included Phillips head screwdriver – **do not** fasten tight until completing step 2. Repeat for second leg.

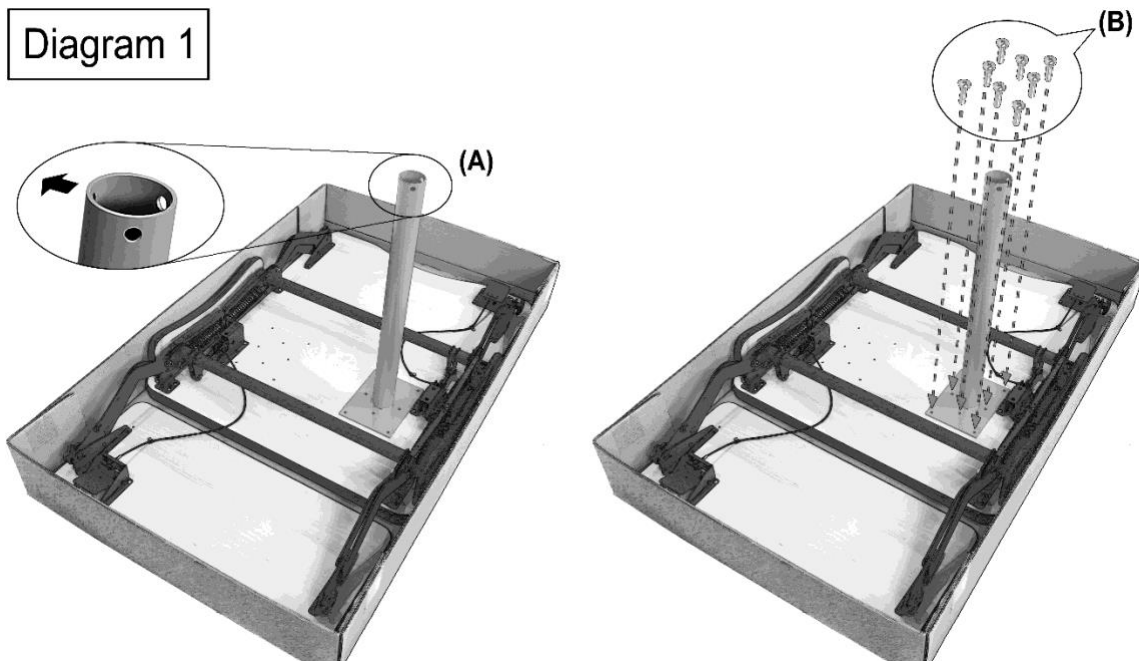


2. Position the spacer bar (D) between the two attached legs, lining up the holes. Attach the spacer bar between the two legs with two - bolt/ bracket/nut combos (E) using the two 9/16th wrenches provided as shown in diagram (2). **⚠ CAUTION** – Overtightening may result in misalignment and/or cause damage to equipment. Once spacer bar is in place and legs are properly aligned, tightly fasten screws (D) to secure each leg to table and finish fastening screws (B) from step 1.

Diagram 2

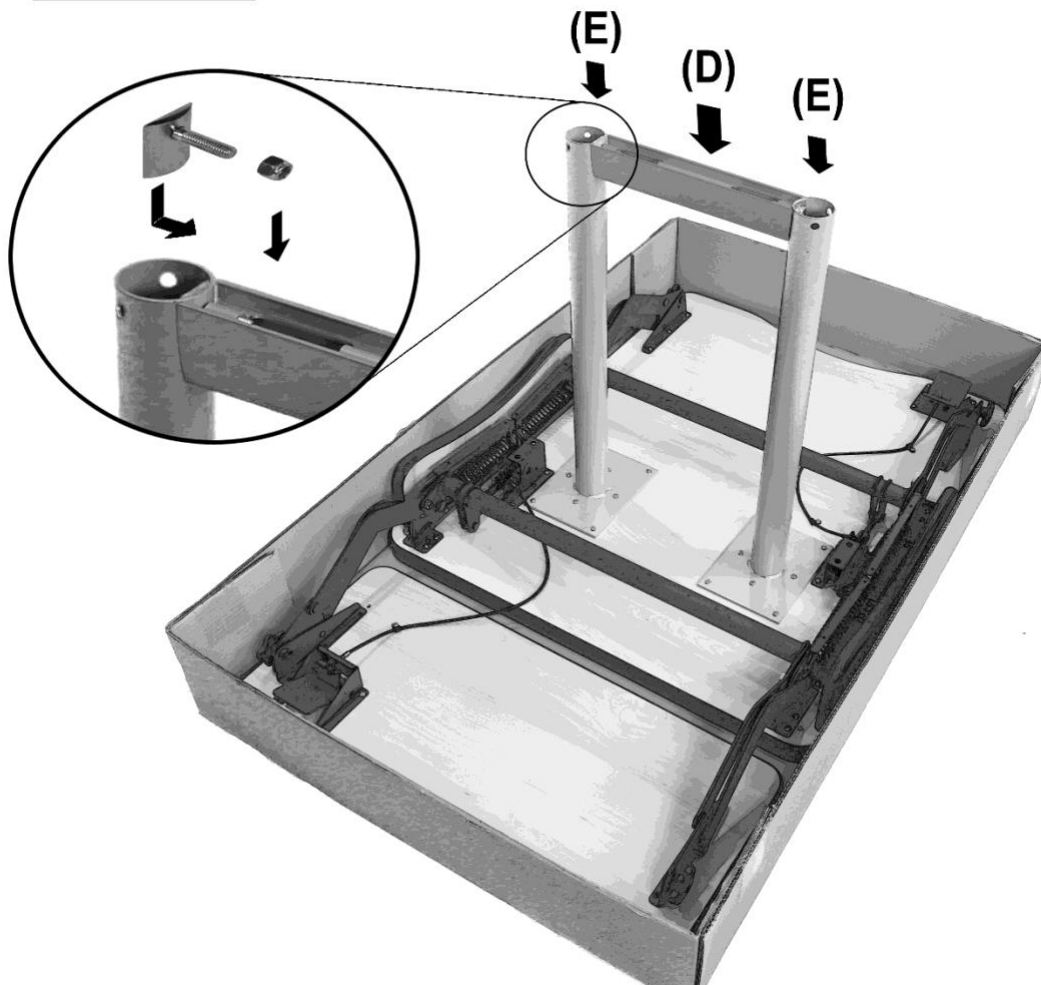


3. Position 1 foot (G) on the outside of one of the mounted legs and tighten with a bolt/bracket/nut combo (E) using the two included 9/16th wrenches as shown as in diagram (3). Ensure each bolt is firmly secure then repeat for second foot. Repeat on opposite side attaching the remaining two feet to the second leg. **⚠ CAUTION** – Overtightening may result in misalignment and/or cause damage to equipment.

Diagram 3

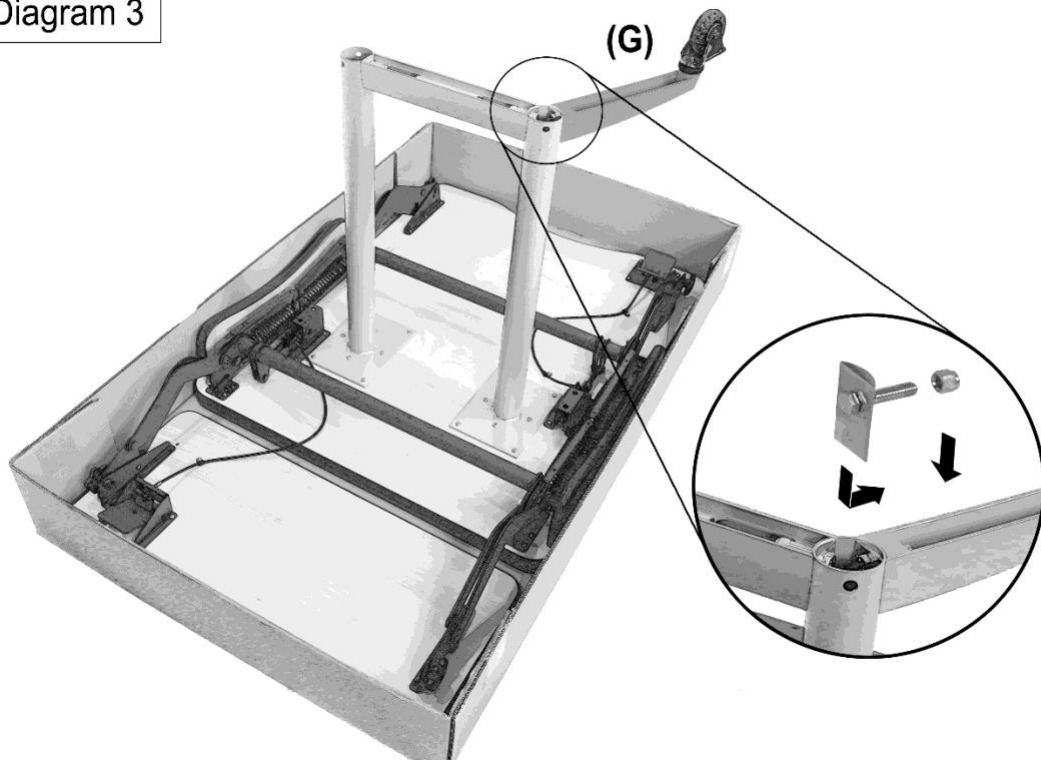


- Using 2 people lay table on side, then stand table upright as shown in diagram (4).

Diagram 4

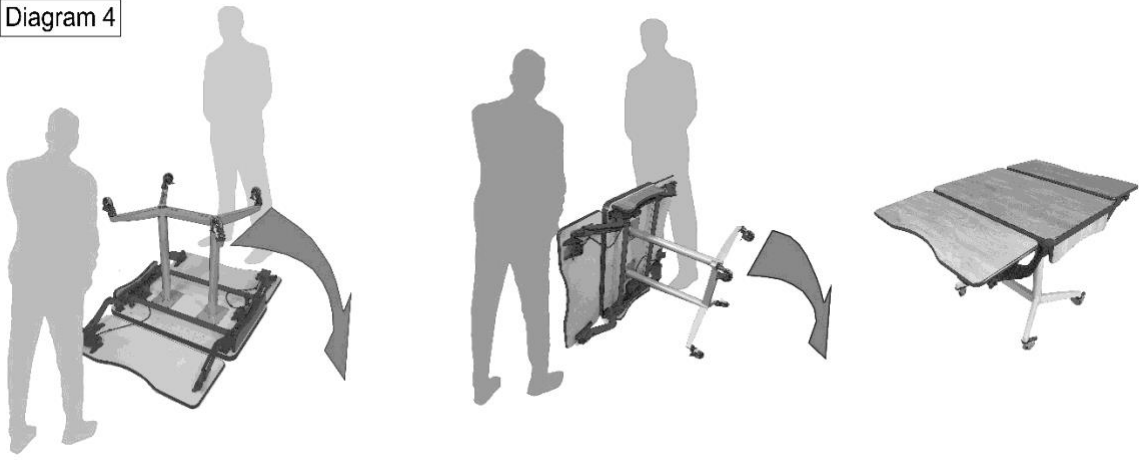

- Using 2 people, have 1 person firmly hold a flex tray in position, while the other cuts the corresponding zip tie with scissors at the handle as shown in diagram (5). **⚠️ WARNING - The flex trays are spring loaded and under tension.** Extreme caution should be taken to ensure the tray is firmly held as it can move up rapidly once the zip tie is removed.

Diagram 5

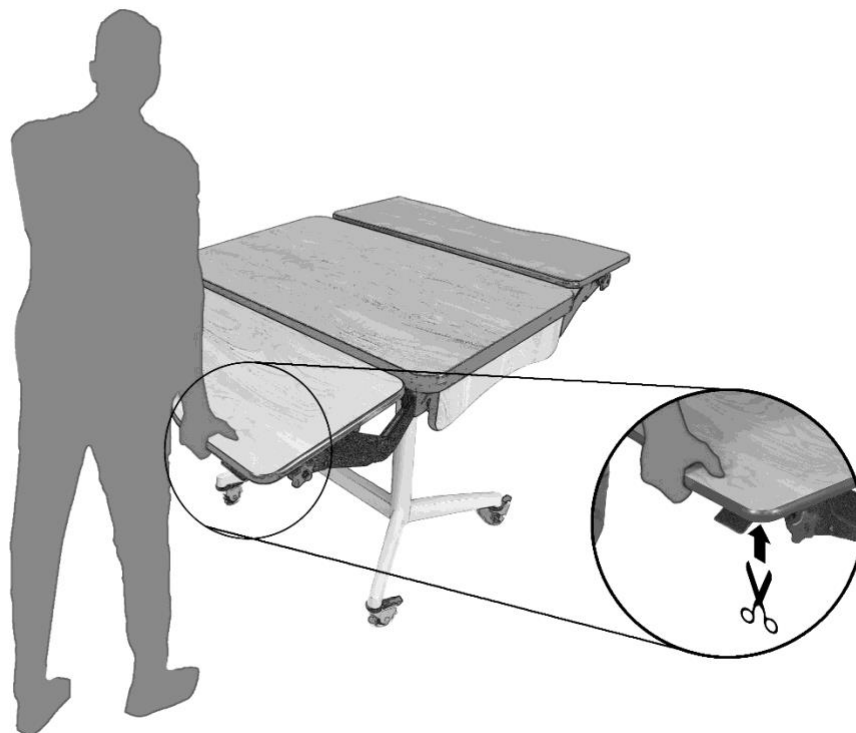

Using your FlexTable®


Intended Use

The FlexTable is a multi-surface table designed and tested to accommodate the different preferred table heights and angles of seated individuals. These accommodations enable individuals of different shapes, sizes and abilities to participate in seated tabletop activities like eating, together.

Adjustments

Moving the flex tray up and down

- All caster locks should be engaged to prevent movement while adjusting.
- Users should be seated at flex tray prior to adjusting.
- Make sure the flex tray mechanism is clear of obstructions before adjusting.
 - **⚠ WARNING** – Obstructing movement of flex tray mechanism can result in damage to the mechanism and/or personal injury.
- To adjust height of tray, firmly hold flex tray while depressing paddle, release paddle once the tray is in the preferred location.

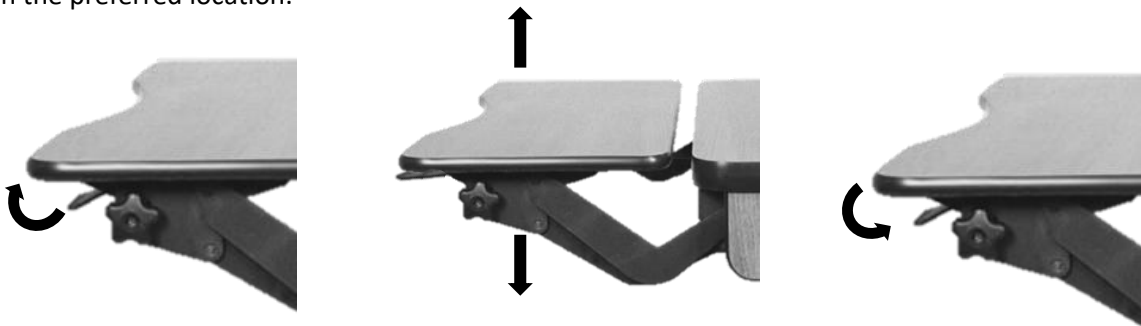

- **⚠ WARNING** - The flex trays are spring loaded and under tension. Trays must be held firmly while adjusting as tray will move up rapidly once paddle is depressed.


Adjusting the angle of you flex tray

- Turn both knobs counter clockwise to release tension, position at desired angle, turn both knobs clockwise to tighten.




Maximum weight capacity

- The FlexTable® top and base can hold no more than 50 lbs.
- Each flex tray can hold no more than 10lbs

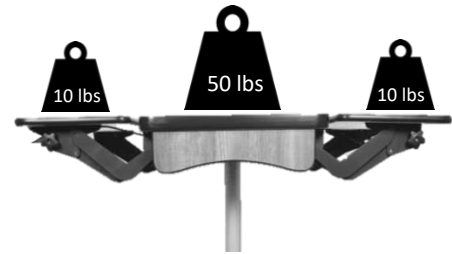

Joining Tables

- Line up tables side by side.
- Ensure joining pins are matched to pins on opposing table.
- Attach joining band over pins on opposing tables.
- Remove joining band to separate.

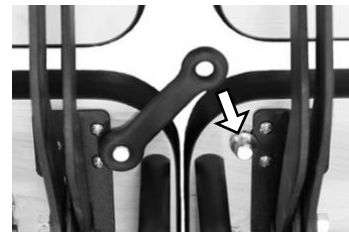

*Joining bands are included when ordering 2 or more FlexTables®. If you require additional bands, please email customer service at: FlexTable@tcf.org

Care and Maintenance

- Clean with disinfectant or soap and water and dry thoroughly using a soft towel or paper towels.
 - **CAUTION:** Never clean the FlexTable® using an abrasive pad as this can scratch and damage surfaces.
- Check and clear casters of debris regularly to ensure smooth movement.
- Engage FlexTable® handle regularly to ensure mechanism is working properly.

WARRANTY

LIMITED PRODUCT WARRANTY

This limited warranty applies to defects in material and workmanship when the product is maintained and used for its intended purpose. If you believe your product is covered by our limited warranty, please contact us at **FlexTable@tcf.org**. We will examine the product and decide in our discretion if it is covered by this limited warranty. If covered, we will either repair the product or replace it, in our sole discretion. If it's determined that the failure was caused by misuse or abuse of the product or otherwise not covered as set forth below, replacements and return shipping will be at the customer's expense. This warranty is intended to benefit the original purchaser of the product in the United States. This warranty does not cover consequential damages.

This limited warranty does not cover:

- Changes in surface finish
- Normal wear and tear
- Modifications made by the user
- Products that were not installed or maintained in accordance with product instructions and warnings
- Damage or marking of surfaces caused by sharp or foreign objects
- Use of the product for an unintended purpose

APPLICABLE WARRANTY PERIODS

2 YEARS from the date of purchase – All surfaces, table tops, and legs/base.

1 YEAR from the date of purchase – Mechanical parts and caster wheels.