USER GUIDE 1-8

GUIA DEL USUARIO 9-16

COMODITÀ

Excellence in design and safety

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COMPACT ROLLING WALKER

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model number: COM 900

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COMODITÀ is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

The modern patented design and sturdy construction of the COMODITÀ Tipo Classic Rolling Walker (ref. COM 900) make it one of the safest and most stylish walkers in the market.

Built to support weights of up to 300 lb, it is tested and approved under international testing standards for walkers and rollators.

The COMODITÀ Tipo Classic Rolling Walker is built with the best materials available to ensure the highest safety you can find.

In addition, the COMODITÀ Tipo Classic Rolling Walker comes with the exclusive COMODITÀ Pisa removable cup holder, cane holder and washable nylon shopping bag, as well as a convenient travel bag.

The COMODITÀ Tipo Classic Rolling Walker has two 10" aluminum front wheels, making it very easy to maneuver. It can also be easily double folded for storage or transportation.

Please do not hesitate to contact us through our website www.comoditaua.com if you have any doubts, questions or problems. We are here to help!

Main Features:

- Extra-large 10" aluminum front wheels for safer indoor and outdoor use
- Wide, comfortable nylon seat
- Adjustable handle height
- Padded ergonomic backrest
- Convenient removable cup holder
- Cane holder included
- Convenient transport nylon bag
- Hand brake locks for safety while sitting
- Double fold construction makes it very compact and easy to transport
- Four light reflectors at the front and two at the back for additional safety
- Nylon shopping bag with light reflecting strap for added security

Product Dimensions and Specifications

- Unit weight: 18½ lb (8.5 kg)
- Weight capacity: 300 lb (136 kg)
- Recommended user height: 5'4" to 6'4" (1,62 m to 1,90 m)
- Overall height: 31½" / 37½" (80.5 cm / 95.5 cm)
- Overall length: 27" (69 cm)
- Overall width: 24" (61cm)
- Folded width: 10½" (27 cm)
- Front wheel diameter: 10" (25 cm)
- Back wheel diameter: 8" (20 cm)
- Seat height from floor: 22½" (57.5 cm)
- Seat depth: 11" (28.5 cm)
- Seat width: 17½" (44 cm)
- Size of nylon bag: 17" W, 8" D, 10" H (43cm W, 20cm D, 25cm H)

Assembling your COMODITÀ Tipo Classic Rolling Walker is extremely simple!

1. Carefully remove the walker from the shipping carton.
2. Place the rollator on a flat surface.

3. Raise the handles to the upright position.

4. Lock the frame on both sides by bringing the folding lock lever (pic 1) up 180 degrees, so it points up (pic 2). Turn the folding lever 90 degrees towards the back wheel, so it slides into the folding lock lever plastic support (pic 3).
Note: You can only slide the folding lock lever into the plastic support if it is in the correct position, with most of the extension of the lever very close to the plastic frame.

PIC 1

PIC 2

PIC 3

5. Release the plastic locking device that holds the walker folded.

6. Open the walker and lock it in the opened position by pushing down the sides of the seat onto the main frame until you hear a click.

7. Insert both sides of the nylon bag into the nylon bag metal support.

8. Insert the cup holder into its plastic support on the side of the walker.

9. Adjust the height of the handles by standing upright behind the walker with the shoulders relaxed and arms hanging at your side. The handgrips should be at a height even with your wrists.

IT IS THAT SIMPLE! NO TOOLS, NO LOOSE PARTS!

Operating Instructions

TO WALK:

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, slowly squeeze the brake handles up.
- To lock the brakes, push down on the brake handles until the brakes “click” into locked position. Do not push it any further than this point.
- To unlock the brakes and resume walking, pull up the brake handles until you hear a “click.”

TO USE THE SEAT OF YOUR WALKER:

- Make sure your walker is in a flat surface, not on an incline.
- LOCK the brakes before sitting.
- Make sure your feet rest flat on the ground.

Care and Maintenance

PERIODIC BRAKE ADJUSTMENT

To adjust the sensitivity of the brake:

- To tighten the brake (reduce the gap between the brake pad and wheel) rotate the adjusting nut clockwise.
- To loosen the brake (increase the gap between the brake pad and the wheel) rotate the adjusting nut counterclockwise.

WARNING!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- **DO NOT USE WALKER IF BRAKES ARE NOT WORKING PROPERLY.**

WARNING!!! Failure to follow safety instructions may result in serious injuries.

- **DO NOT** use this product as a wheelchair.
- **DO NOT** transport someone while sitting on this walker.
- **DO NOT** exceed maximum weight capacity.
- Brakes **MUST** be on **LOCKED** position before sitting on the seat.
- **DO NOT** self-propel the walker while seated.
- **DO NOT** sit on the walker on an incline.
- **INSPECT BRAKES** before each use.
- **DO NOT** use the walker if the brakes are not working properly.
- **ALL WHEELS MUST** be in contact with the floor or ground at all times.
- **BACKREST IS NOT** intended to support your full weight. It is intended to provide back support while you are seated.
- **DO NOT** use in escalators, stairs, moving sidewalks or uneven surfaces.
- **BOTH FEET** should rest flat on the ground when sitting.
- **ALWAYS** check if the seat is locked down and well connected to the frame when the rollator is open.

Warranty

Your Comodità Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase for the original owner. This warranty does not cover product failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our discretion, repair or replace the product. This warranty does not include any labor or shipping charges. Please contact our customer service at info@comoditausa.com. We will provide a return authorization number. You will be required to include a copy of your original bill of sale.

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COMODITÀ LLC USA