

ACCESS SLING




FITTING INSTRUCTIONS
TECHNICAL SPECIFICATIONS
WASHING INSTRUCTIONS



USER GUIDE FOR THE ACCESS SLING

The Access Sling has been designed specifically to facilitate the toileting process. It is an easy to fit sling, padded for additional comfort which allows access to and removal of clothing. It is not a General Purpose sling and will only suit approx 25% of clients.

Advice is available from our authorised dealers.

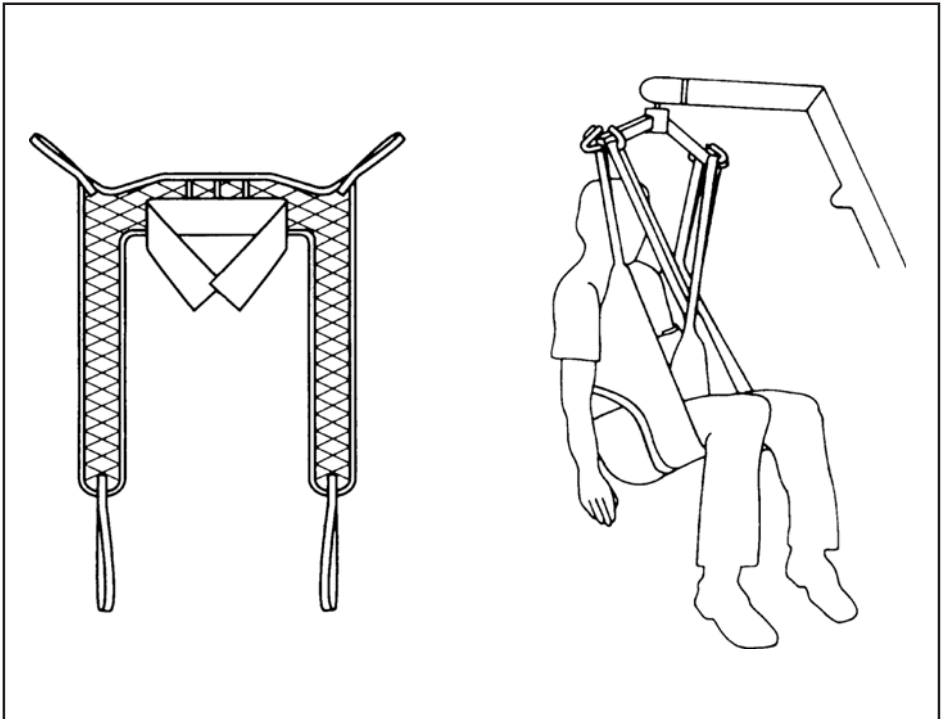

ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN
AUTHORISED DEALER BEFORE PURCHASING.

WE WILL BE PLEASED TO ADVISE YOU AS TO YOUR NEAREST AUTHORISED DEALER.




HOW TO FIT FROM SEATED POSITION




Place the sling down the back to position just beneath the shoulder blades and above the belt line. Attach Velcro strap around the front of the client. There is no need for this to be tight.



Raise the client's leg and feed the leg strap under and up between the legs. Ensure that the sling is not twisted or creased under the thigh. Repeat this procedure for the other leg.

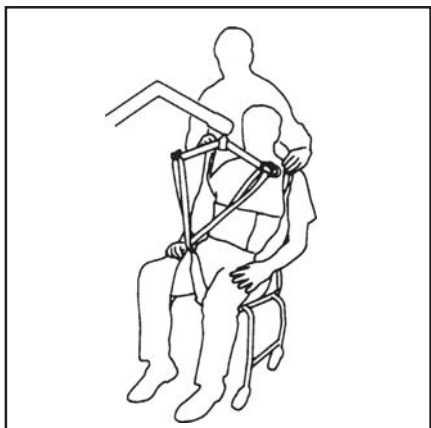


Cross over leg straps and attach to the hoist on the front hooks at longest loop. Raise the hoist and attach shoulder loops at the rear hook (a more upright position will be achieved at shortest loop). The hoist may move towards the client.

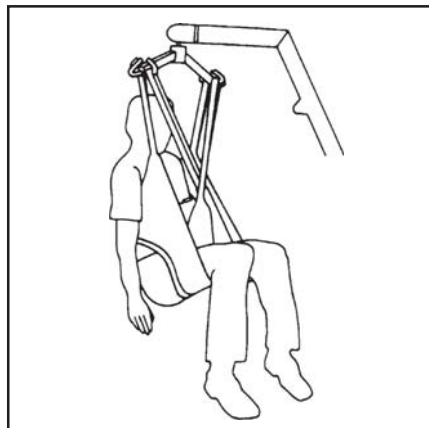

Raise the hoist to the required height. When reseating the client, light pressure on the thigh will ensure a good posture.

ALTERNATIVE SEATED POSITION

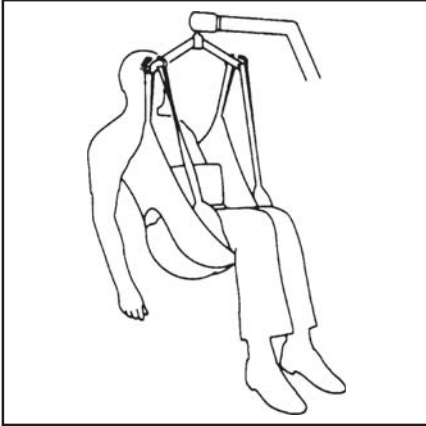

For closed leg positions, cross over straps under both legs and attach to hoist as shown. This can provide more comfort.



Some clients may be comfortable with the leg pieces brought down between the thighs and then taken around the outside of the legs as shown.

HOW TO FIT FROM LYING POSITION


The sling is not recommended for use in this situation.

ACCESS SLING TECHNICAL SPECIFICATIONS

STANDARD	POLYESTER BS 5827
CERTIFICATE OF CONFORMITY	QUAD 151 ISSUE 1
SAFE WORKING LOAD	460lbs / 210kg

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

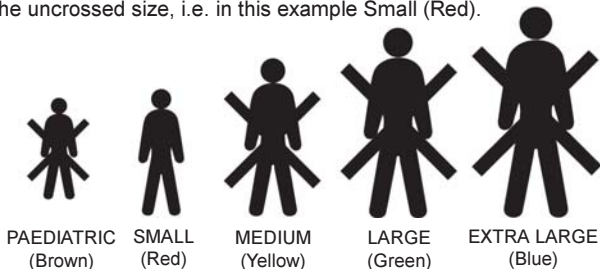

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



		US	UK	
ORDER CODE	PAEDIATRIC (BROWN)	NA	SL	1600 (Polyester)
	SMALL (RED)	NA	SL	1601 (Polyester)
	MEDIUM (YELLOW)	NA	SL	1603 (Polyester)
	LARGE (GREEN)	NA	SL	1606 (Polyester)
	EXTRA LARGE (BLUE)	NA	SL	1608 (Polyester)



Sunrise Medical Inc.
 7477 East Dry Creek Parkway
 Longmont CO 80503. USA
 Tel: (303) 218 4600
 (800) 333 4000
www.sunrisemedical.com



Sunrise Medical Limited
 High Street Wollaston
 West Midlands DY8 4PS England
 Tel: 01384 44 66 88
 Fax: 01384 44 66 99
www.sunrisemedical.com