


User Instruction Manual

Hoyer® One Piece Sling

To avoid injury, read user manual prior to use.

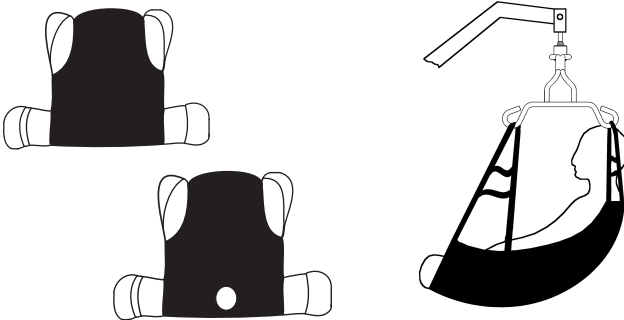


⚠ WARNING

- **JOERNS RECOMMENDS THE USE OF GENUINE HOYER SLINGS.** Hoyer brand slings are approved by Joerns for use on Joerns brand lifts. Slings used on Joerns brand lifts, including Hoyer lifts, must be approved by Joerns.
- For the safety of the patient and carer; before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.
- **CHECK** sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.
- **DO NOT** alter slings. Destroy and discard worn slings.
- **NEVER** leave a patient unattended.
- **DO NOT** exceed the rated capacity of the sling or lift.
- **DO NOT** attempt to re-position a patient by pulling on the sling loops.


Introduction

The One Piece Sling is designed for amputees but it is also suitable for other patients who have to remain in the sling for an extended period, i.e. during bed making or when bathing. It is suitable for lifting amputees, but patients must be individually assessed for suitability by a qualified person. Advice is available from our authorized dealers or Joerns Healthcare.

How to Use the One Piece Sling

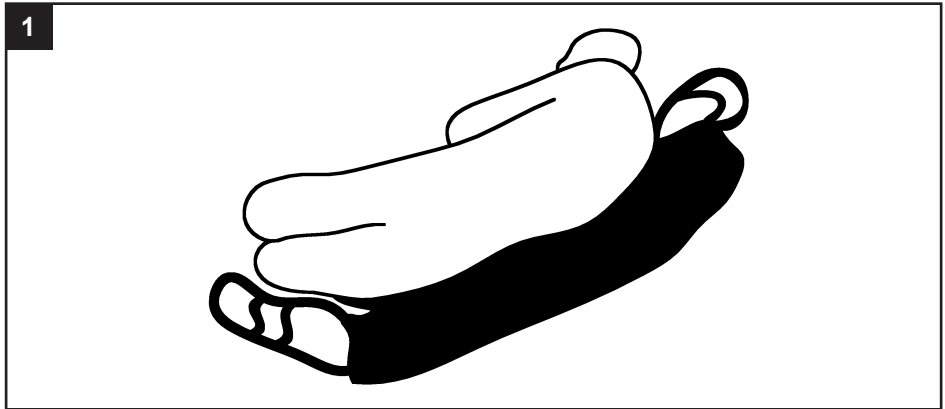

From the recumbent position, roll the patient onto the sling using the draw sheet roll. Ensure the bottom of the sling is just above the knee area.

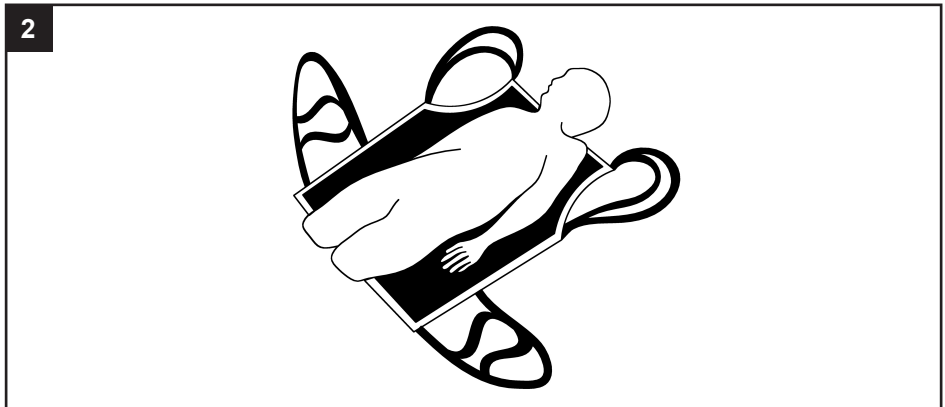

Ensure the sling is laid flat underneath the patient. The top of the sling should be at shoulder blade height and also level across the top and bottom edges.


How to Use the One Piece Sling

3



Move the spreader bar into position above the patient. Attach the shoulder strap and leg loops to the respective front and rear hooks on the cradle.

4

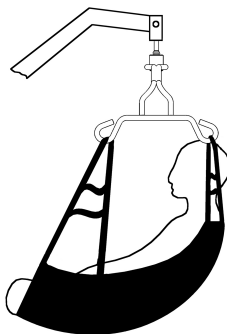

Raise the lift to the required height. NOTE: It may be necessary to support the head while the patient is suspended.

Joerns Healthcare recommends that slings are checked regularly and before each use, for fraying or damage.

Technical Specifications

STANDARDS	EN ISO 10535, CE
SAFE WORKING LOAD	600lbs / 272kg

Sizing & Part Numbers

SIZE	One Piece with Strap	One Piece Commode with Strap
SMALL	70053	70058
MEDIUM	70052	70057
		70057SE
LARGE	70051	70056
		70056SE
X LARGE	SPECIAL	70055

Washing Instructions



Machine wash at 185°F.



DO NOT wash with bleach. Bleach will damage the sling's material and make it unsafe for use.



Cool tumble dry, air dry or dry at very low temperature.



DO NOT dry clean.

⚠ WARNING

Slings can suffer damage during washing and drying and should be checked carefully before each use.

Additional slings are available to suit individual needs. Contact your Joerns representative for more information.