*NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use. Contact Carrier/Dealer for further instructions.*

### SAFETY SUMMARY

The following recommendations are made for the safe installation and use of the Seat Rigidizer:

#### WARNING

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Ensure that Seat Rigidizer is properly installed per instructions. Movement of the Rigidizer within the cushion may occur causing discomfort to the user if not properly installed.

DO NOT use the Seat Rigidizer on a wheelchair equipped with a rigid seat pan. Use only on sling seat wheelchairs.

The Seat Rigidizer does not in any way change the weight limitation of your wheelchair.

### INSTALLING THE SEAT RIGIDIZER (FIGURE 1)

*NOTE: To change the depth of the seat rigidizer, refer to **CUSTOMIZING THE SEAT RIGIDIZER** in this instruction sheet.*

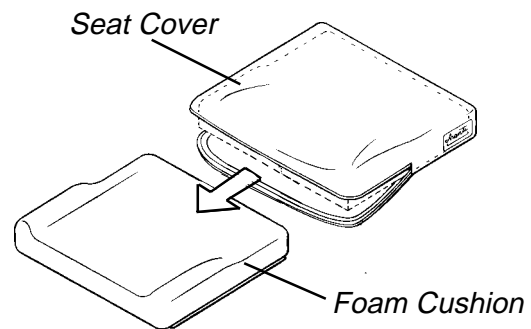
1. Remove seat cushion assembly from chair.
2. Open seat cover and remove the foam cushion from seat cover.

*NOTE: DO NOT REMOVE the backing from double-sided tape until you have marked the placement area on the foam cushion. Be sure of placement area as the rigidizer will not re-adhere securely if installed and removed.*


3. Position bottom (**underside**) of the Seat Rigidizer onto the bottom of the foam cushion to the extreme rear while centering it on the left and right sides (**FIGURE 1**).
4. Outline the foam cushion with a marker, pen or pencil using the Seat Rigidizer as your template to mark the placement area.
5. After marking placement area, remove the double-sided tape backing from Seat Rigidizer and install Rigidizer onto bottom (**underside**) of foam cushion.

6. Press down firmly to adhere the Seat Rigidizer to the foam cushion.
7. Re-install foam cushion into the seat cover and secure seat cover to chair with the hook and loop backing.

#### REMOVE FOAM CUSHION FROM SEAT COVER STEPS 1-2

#### INSTALL RIGIDIZER TO FOAM CUSHION STEPS 3-6
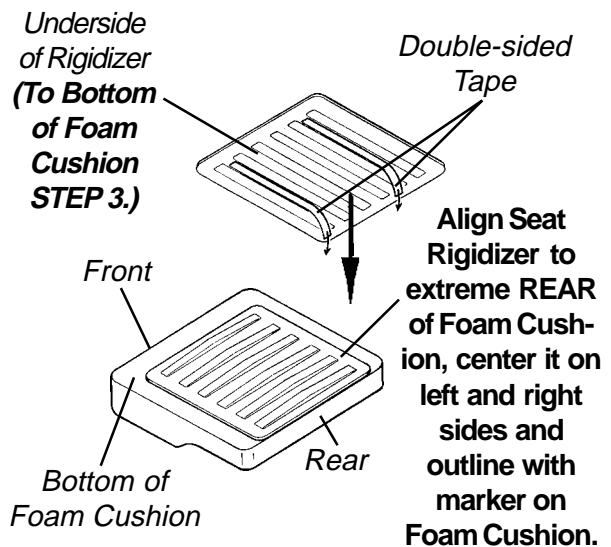
#### REINSTALL CUSHION INTO SEAT COVER - STEP 7

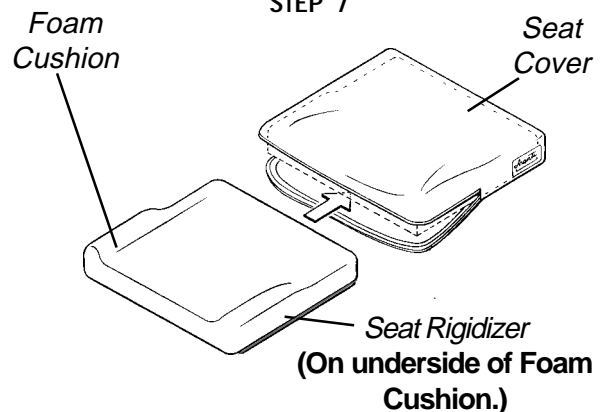
FIGURE 1 - INSTALLING THE SEAT RIGIDIZER

## CUSTOMIZING THE SEAT RIGIDIZER (FIGURE 2)

The Seat Rigidizer is available in various widths and depths and can be trimmed to fit shorter seat depths. WIDTH and DEPTH sizes equal ACTUAL size of the rigidizer to be used for listed CUSHION SIZE Width X Depth.

15-inch (38 cm) deep Rigidizers can be trimmed 2-inches (5 cm) to fit 14-inch (36 cm) cushion depths, as shown in FIGURE 2.

### CAUTION

When trimming the rigidizer, ensure the cut edges are smooth to reduce wear on the inner cover of the cushion.

CUSHION SIZE Width x Depth (cm)	RIGIDIZER SIZE Width x Depth (cm)	TRIM 5 cm from Depth
41 X 40	40 X 40	NO
43 X 40		
41 X 41	40 X 40	NO
43 X 43		
41 X 45	40 X 45	NO
43 X 45		
41 X 46	40 X 45	NO
43 X 48		
46 X 40	45 X 40	NO
48 X 40		
46 X 43	45 X 45	NO
46 X 45		
48 X 45	45 X 45	NO
46 X 46		
46 X 48	50 X 40	NO
50 X 40		
55 X 43	50 X 45	NO
50 X 45		
55 X 45	50 X 45	NO
51 X 46		
55 X 47		

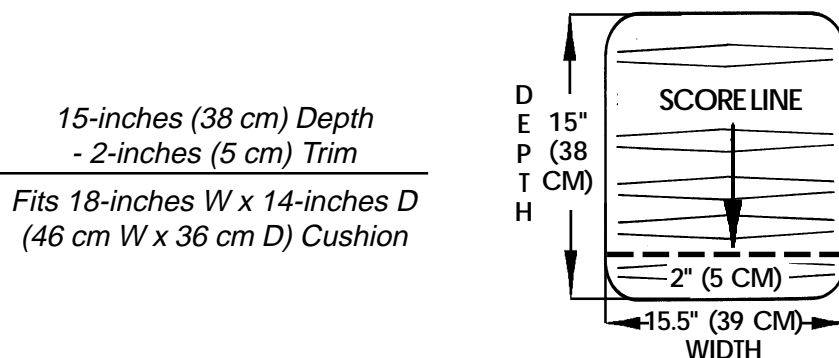
FIGURE 2 - CUSTOMIZING THE SEAT RIGIDIZER - SAMPLE TRIM LINES AND DIMENSIONS

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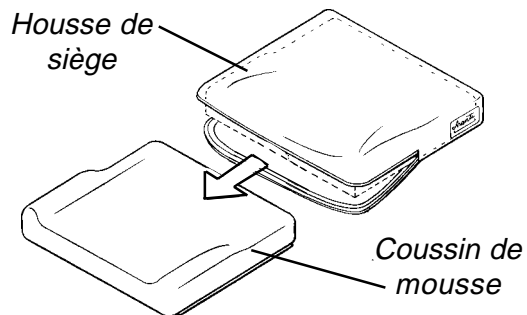
### INSTALLATION DU COUSSIN RIGIDIZER (FIGURE 1)

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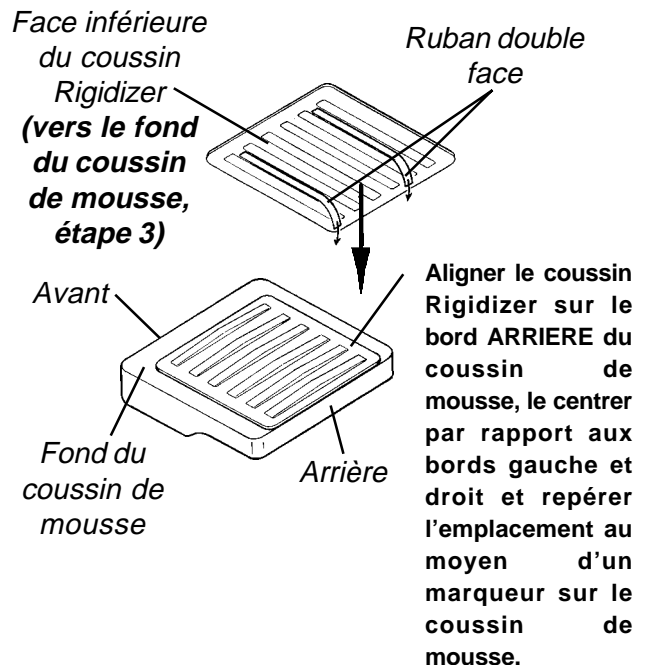
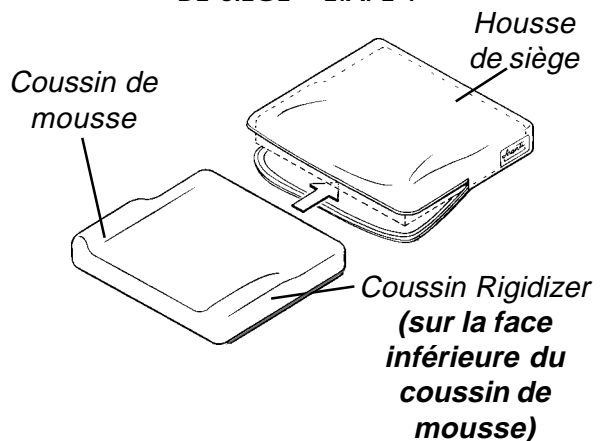
FIGURE 1 - INSTALLATION DU COUSSIN RIGIDIZER

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© 2000 Invacare Corporation
Invacare is a trademarks of Invacare Corporation.
Finland: Invacare/Scandinavian Mobility AB
Schweiz: Kuschall Design AG
Invacare Corporation [www.invacare.com](http://www.invacare.com)
**REA Stolen AB**