

## NuDrive wheel size compatibility

NuDrive is compatible with most spoked wheelchair wheels of 22" and 24" sizes and some wheels of 25" sizes.

To determine if your wheels are compatible with the NuDrive there are two things that are important.

1. You must measure the inner diameter of your wheel. In the table you can read the possible inner diameter range the NuDrive can fit into depending on which of the three different sets of adapters, that is supplied with the NuDrive, is being used.



Wheel size	Adapter #1	Adapter #2	Adapter #3
NuDrive 22" version	$\varnothing = 47,40 - 48,90$ cm	$\varnothing = 46,40 - 47,90$ cm	$\varnothing = 46,40 - 47,90$ cm
	$\varnothing = 18,68'' - 19,25''$	$\varnothing = 18,27'' - 18,86''$	$\varnothing = 18,27'' - 18,86''$
NuDrive 24"/25" version	$\varnothing = 52,50 - 54,00$ cm	$\varnothing = 51,50 - 53,00$ cm	$\varnothing = 51,50 - 53,00$ cm
	$\varnothing = 20,67'' - 21,26''$	$\varnothing = 20,28'' - 20,87''$	$\varnothing = 20,28'' - 20,87''$

2. You must determine if your rim shape and width are compatible with any of the adapters 1, 2 or 3.



Adapter #1: Suitable for flat rims 22-24 mm wide (0,86" – 0,94")

Adapter #2: Suitable for high profile rims 21-23 mm wide (0,82" – 0,90")

Adapter #3: Suitable for high profile rims 18-20 mm wide (0,71" – 0,78")

Note! For 25" wheels you will most likely need to use #1, even if your wheel has a high profile.



**NOTE!** Your wheel must have less than 36 spokes. MAG wheels are NOT compatible with the NuDrive.