drive™

www.drivemedical.com

Rev.4.04.28.14

folding steel commode
inodoro de acero
plegable

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
item # RTL11148KDR

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Eiland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive™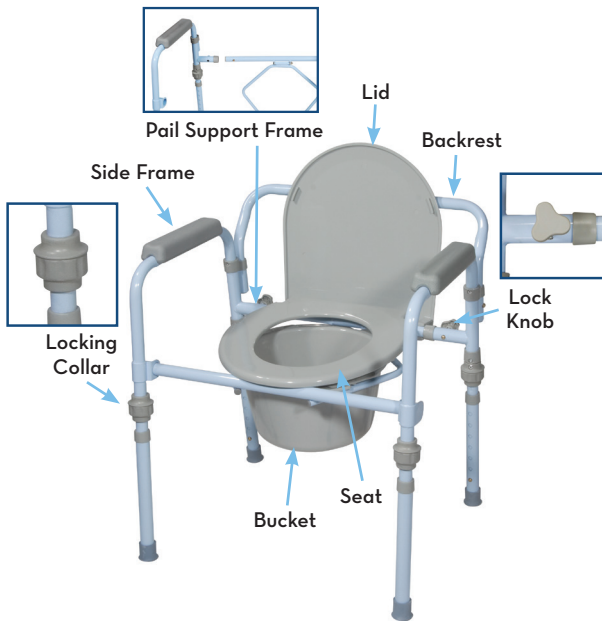

Weight Capacity: 350 lbs (158 Kg)

assembly instructions

1. Remove contents from carton.
2. The legs will automatically slide into the side frame by use of the elastic cords.

Important Note:

The bottom of the side frame (figure 1) and the top of the commode leg feature a mating flange. Match up the flange on the side frame with the flange on the commode leg. Slide the leg up and into the side frame (figure 1). Once the leg is properly fit, slide gray locking collar into the fitting on the side frame and turn clockwise until tight (figure 2). Repeat the process on all legs.

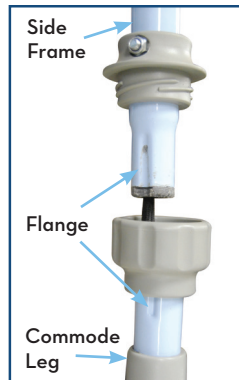
figure 1

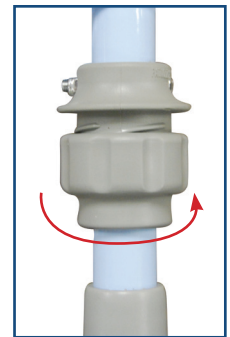
figure 2

3. Line up the side frame with the pail support frame (figure 3). Slide the pail support frame into the side frame while depressing the push pins until the push pin "pops" into the hole on the side frame. Once the one side of the pail support frame is secure in the side frame, turn the lock knob on the back of the side frame until hand tight (figure 4). Repeat on the other side.

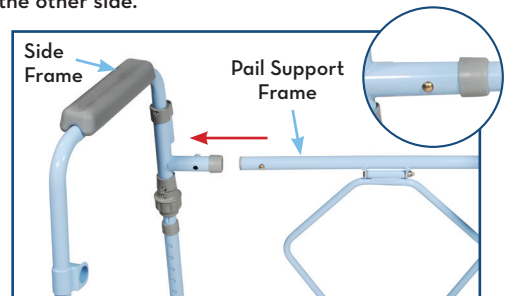
figure 3



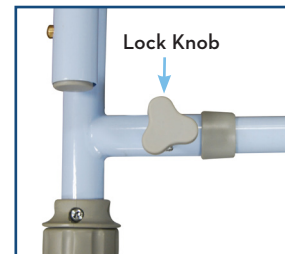
figure 4

- Attach the front of the pail support frame into the side frame as shown in (figure 5). Be sure the push pins appear through the side frame hole. Repeat on the other side.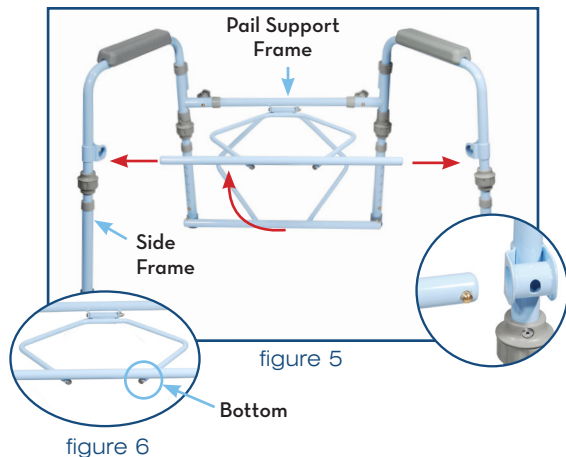

Important Note: Be sure the pail support frame is on the underside of the cross brace (figure 6).

- Attach the backrest by depressing both push pins and sliding the back support into the frame receptacle (figure 7).

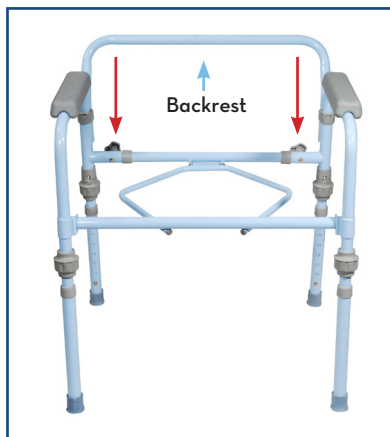
- Lower commode bucket into frame and attach seat and lid onto the back of the frame (figure 8).

figure 8

adjusting leg height

- Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration to the right.
- Be sure legs are locked into place by pushing down on or by pulling legs.

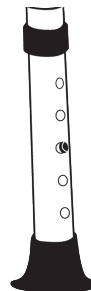additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction manual. If you cannot understand these instructions, then contact a healthcare dealer or technical professional before attempting to install; Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Make certain that the height adjusting snap buttons fully protrude through the same height hole in each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
7. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
8. Users with limited physical strength should be supervised or assisted while using commode.

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington, N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.