

CARE CLINER COLLECTION

OWNERS OPERATING AND MAINTENANCE MANUAL



Care Cliner & XL Care Cliner



(6530 Shown)

Drop-Arm & XL Drop-Arm Care Cliner



(6550 Shown)

Designer Care Cliner



(6710 Shown)

Elite & XL Elite Care Cliner



(6910 Shown)

Swing-Arm & XL Swing-Arm Care Cliner



(6940 Shown)

PLEASE DO NOT DISCARD THIS MANUAL! KEEP FOR FUTURE REFERENCE AND TRAINING

TABLE OF CONTENTS

IMPORTANT SAFETY INFORMATION	4
BACK ASSEMBLY INSTRUCTIONS.....	7
OPERATING INSTRUCTIONS	8
GENERAL MAINTENANCE.....	13
SPECIFICATIONS.....	15
HEAT & MASSAGE OPERATING INSTRUCTIONS	17
CARE CLINER OPTIONS.....	18
WARRANTY INFORMATION.....	19

UNPACKING

1. Carefully examine your product for any damage. Inspect all your components. **IF DAMAGE IS EVIDENT, CONTACT THE FREIGHT CARRIER OR WINCO IMMEDIATELY.**
2. Remove all packing material and hardware that was secured for shipping.
3. Carefully remove all of the components from the carton.
4. Save all boxes and packing material until after you have assembled your product and verified that all components are functioning properly.

IF DAMAGE IS EVIDENT, CONTACT THE FREIGHT CARRIER OR WINCO IMMEDIATELY!

WINCO ASSUMES NO RESPONSIBILITY FOR DAMAGE OR INJURY CAUSED BY THE IMPROPER ASSEMBLY, INSTALLATION, USE OR MAINTENANCE OF THESE PRODUCTS!




IMPORTANT!

Please read this entire manual before using your chair. Do NOT install, maintain or operate this equipment without reading and following this manual otherwise, injury and/or damage may result.

The information contained in this manual is subject to change without notice. Save these instructions for future reference!

No part of this manual may be duplicated in any form without the prior consent of Winco Mfg., LLC. Unauthorized duplication / distribution of these materials may result in civil prosecution to the maximum extent allowed by law.

Products with the following markings are intended for commercial use only:

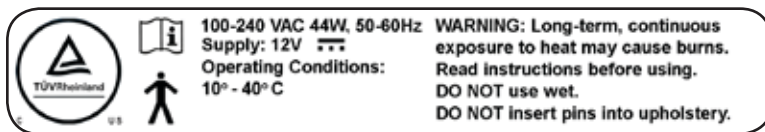

SYMBOLS



FOLLOW INSTRUCTIONS



SYMBOL FOR MANUFACTURER



TYPE B APPLIED



DC VOLTAGE



GENERAL WARNING SIGN



IMPORTANT SAFETY INFORMATION

SAVE THESE INSTRUCTIONS: READ ALL INSTRUCTIONS BEFORE USING PRODUCT FOR PRODUCTS WITH POWERED OPTIONS (HEAT, HEAT & MASSAGE, ETC.)

When using an electrical appliance, basic precautions should always be followed, including the following:

DANGER: To reduce the risk of electric shock always unplug this product from the electrical outlet before cleaning, maintenance or servicing.

WARNING:

1. Unplug this product from the electrical outlet when not in use.
2. Supervision should be provided when using this product.
3. Use this product only for its intended use as described in this manual.
4. Do not use attachments not recommended by the manufacturer.
5. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, contact an authorized Winco service center for instructions.
6. **DO NOT** carry or pull this product by power supply cord or use the cord as a handle.
7. Keep the cord away from heated surfaces.
8. Do not use outdoors.
9. To disconnect, turn all controls to the "off" position, then remove plug from outlet. Pull plug only to remove cord.
10. Clean switch with a damp cloth only – Warranty will be **VOID** if liquid is introduced into switch mechanism
11. Extension cords are not recommended.
12. **DO NOT** roll recliner or other equipment over cord.
13. This product is for use on a nominal 120-volt circuit and has a three-pin plug. If the plug does not fit into the outlet, contact a certified electrician to install the proper outlet. **DO NOT MODIFY THE PLUG OR USE AN ADAPTER WITH THIS PRODUCT.**
14. For an added level of safety; Winco recommends the use of a 120 volt GFCI (ground fault circuit interrupter) outlet.

DANGER:


1. Never insert pins, or attach other metallic fasteners into/onto any part of this chair.
2. Do not use this product if the covering shows signs of deterioration, such as checking, blistering, or cracking
3. **KEEP DRY** - Do not operate in a wet or moist condition or environment.

WARNING:


1. Long term, continuous exposure to heat may cause burns. It is recommended that skin in contact with heated area be checked for redness and blistering during long term usage. Do not use on an infant, invalid or incapacitated person. Do not use on insensitive skin or on a person with poor blood circulation. If you are uncertain if you should use this product, please consult your physician about the use of heat and massage
2. Do Not Crush/Pinch heating elements or wiring.

IMPORTANT: PLEASE READ

For Heat & Heat and Massage Chairs: This equipment is marked with the industry recognized symbol indicating it is a “Class II Double-Insulated” device as defined by IEC 60601-1. As such, it does NOT require a three conductor power cord (i.e. one with a ground connection) for safety purposes. While the equipment can be safely operated with no ground connection, a three conductor power cord is provided to satisfy customer expectations for typical equipment configuration. Devices qualifying as double insulated require different testing & examination procedures than “Class I Grounded” devices. If your facility requires the product to be tested, be sure to follow testing & examination procedures for “Class II Double-Insulated” devices.

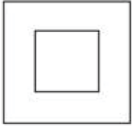

ISOLATION DIAGRAM FOR HEAT & MASSAGE SYSTEM

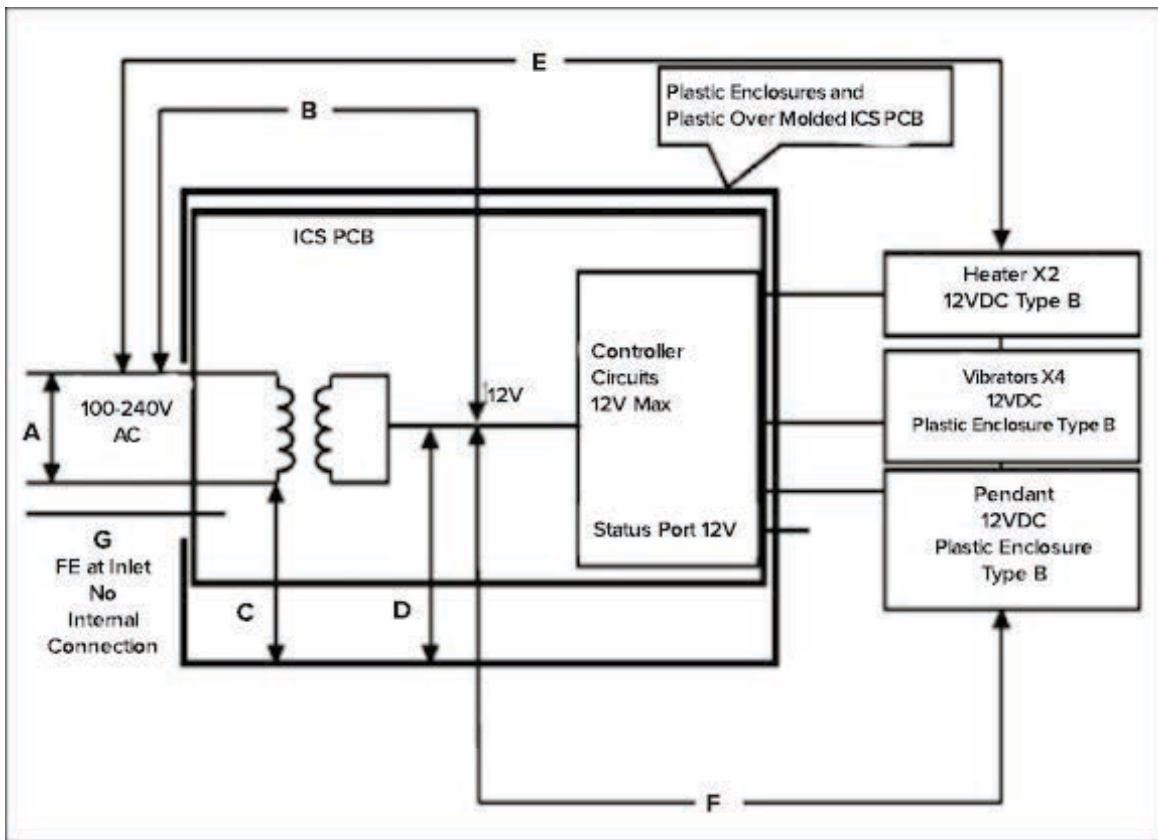

AREA	INSULATION TYPE
A	Basic Operational
B	Double Insulated / Reinforced
C	Double Insulated / Reinforced
D	Double Insulated / Reinforced
E	Double Insulated / Reinforced
F	Double Insulated / Reinforced
G	Inlet FE Pin only connects to inlet & pad



WARNING

IMPORTANT: PLEASE READ

DO NOT LIFT OR CHANGE CHAIR POSITION BY USING THE LEGREST.

THIS COULD CAUSE INJURY TO THE USER OR DAMAGE TO THE RECLINE MECHANISM AND WILL VOID WARRANTY.




ALWAYS CHANGE POSITION FROM BACK OF CHAIR.

1. **READ AND FOLLOW ALL DIRECTIONS.**
2. Keep children away from extended foot support (or other similar parts).
3. **DO NOT** put hands, feet or clothing into any openings when changing positions on recliner. Attendant **MUST** confirm that users arms, legs, hands and feet are safe while changing recliner positions or **SERIOUS INJURY MAY RESULT.**
4. Remove or move chair accessories out of the way of patients legs (ie. over-lap table & pivot table) when reclining/changing position of patient.
5. **LOCK** casters at all times when chair is stationary. **UNLOCK** casters when moving/transporting chair.
6. **STAY CLEAR** of the recliner mechanisms.
7. **DO NOT** use recliner for transporting in or with **ANY** type of vehicle or trailer. Winco recliners have not been tested or approved for use by an occupant in any type of vehicle or trailer.
8. **NEVER** use the chair arms or backrest as a seat; **SERIOUS INJURY** or damage may occur.
9. Periodically check the tightness of all nuts, bolts and screws.
10. Immediately **REMOVE FROM SERVICE**; Any recliner with broken recline mechanisms, torn upholstery, or other mechanical or visible damage.
11. **NEVER** transport user with chair arms in an "open or down" position. Applies to Drop-Arm & Swing-Arm models.
12. **USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.**
13. **NEVER EXCEED** the recommended weight capacity (see page 15-16).

BACK ASSEMBLY INSTRUCTIONS

1. Remove back assembly and chair frame from carton.
NOTE: Heat Only and Heat & Massage chairs, **AVOID** disconnecting any "chair to back" wires.
NOTE: Make sure the footrest is in closed position before installing the back.
2. Lock all casters on chair base. (see Lock/Unlock Casters page 8)
3. Remove **ALL** sets of clevis & bow-tie cotter pins located on the bottom L&R Forks of the Back Frame and on the operator of the gas cylinder. **DO NOT LOSE THESE PARTS, YOU WILL NEED THEM AGAIN! (FIG. 1)**
4. Position the Back Frame Forks over the Seat Frame Rails and align holes **(FIG 1)**.
HELPFUL HINT: Align holes on Lower Back Frame while allowing the Upholstered Back to rest on the seat; This will help you steady the chair back. **If possible have someone help you hold the back assembly while you install pins**
5. Align & insert clevis pins (removed in step 4) through each Back Frame Fork and corresponding hole in the Seat Frame.
 - The small hole in the clevis pin **MUST** be to the **INSIDE** of the chair. **(FIG.2)**
 - » **MODELS 653 & 654 ONLY:** The clevis pin can only be installed with the small pin hole to the **OUTSIDE** of the chair.
 - » **MODELS w/ HEAT or HEAT& MASSAGE:** The wiring from the back should be positioned between the back frame and the upholstered seat. **(FIG.2A) (DO NOT INSTALL WIRES BETWEEN BACK FRAME FORKS)**
 - Make sure pin goes through holes in seat frame tube, **NOT UNDER SEAT FRAME TUBE.**
 - Insert bow-tie cotter pins into small hole of each clevis pin. **(FIG.2B)**
6. Align Operator holes between the Seat Frame bracket holes on Seat Frame assembly. Install the saved clevis and bow-tie cotter pin through the aligned holes. **(FIG. 3)**

Make sure all bow tie cotter pins are securely inserted in all clevis pins and that cable end is attached through slot of Operator. Failure to securely insert the bow tie cotter pins into the clevis pin could cause the clevis pins to fall out and result in injury or damage to the chair.

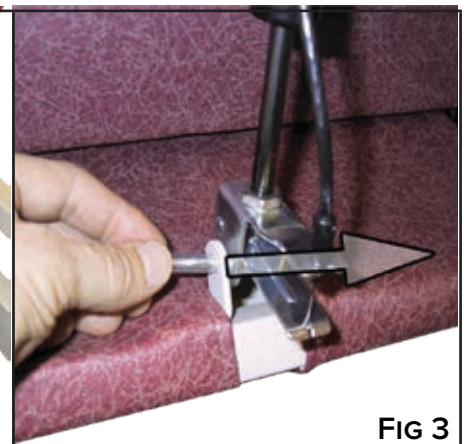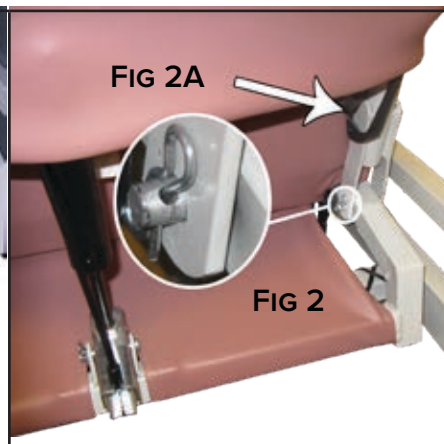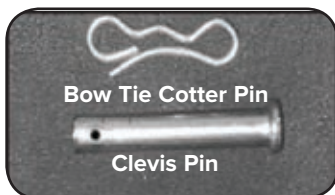


OPERATING INSTRUCTIONS

RECLINING THE CARE CLINER

ALL CHAIRS HAVE 3 STANDARD POSITIONS PLUS TRENDELENBURG POSITIONING ON SELECT MODELS* (*WHEN GAS SPRING IS ENGAGED WITH RELEASE HANDLES FROM 3RD RECLINE POSITION)

- From the #1 (SEATED UPRIGHT) position, grip the arm rests and simultaneously lean body weight back. Push rearward with your arms until desired recline position #2 (TV) or #3 (RECLINE) is reached. **(FIG.1)**
 - NOTE:** The footrest will simultaneously elevate.
- An attendant can also recline the chair to position #2 or #3 by simply grasping the armrest with one hand, AND the top of the chair back with the other; Then push back on the back of the chair. **(FIG.2)**
 - NOTE:** This applies ONLY to positions 2 & 3, and **NOT** engaging the gas spring for trendelenburg.
- TRENDELENBURG POSITIONING MUST ONLY BE PERFORMED BY AN ATTENDANT.** While occupant is in the chair, place the chair into the fully reclined position(#3). The gas spring **MUST** be engaged by squeezing the Release Handle at the top of the backrest. **(FIG. 3 & 4)**
 - DO NOT FORCE** back of chair downward without squeezing the Release Handle. This will damage the chair . The chair back will stay in trendelenburg position when pressure is removed from the Release Handle.
- After each use of Trendelenburg return the chair back to regular positioning by squeezing the Release Handle and lifting up until chair back is fully upright.
 - IMPORTANT:** Attendant must return the chair back to the full upright position after each use when patient exits the chair and before another patient enters chair.

 **NOTE:** IF THE CHAIR BACK DOES NOT RECLINE WHEN THE RELEASE BAR IS SQUEEZED, OR DOES NOT REMAIN IN CORRECT RECLINE POSITION WHEN THE RELEASE HANDLE IS RELEASED, CABLE TENSION MAY NEED TO BE ADJUSTED. REFER TO ADJUSTING GAS SPRING ON PAGE 11.

NEVER PUSH OR PULL FOOTREST TO POSITION CHAIR.

NEVER STAND OR SIT ON THE FOOT REST OR LEG REST.

NEVER PLACE HANDS, LEGS, OR FEET UNDER SEAT, LEG REST, OR FOOT REST. STAY CLEAR OF RECLINE MECHANISM.

KEEP CHILDREN AWAY FROM UNDERSIDE OF CHAIR. INJURY MAY OCCUR.




FIG 1



FIG 2



FIG 3

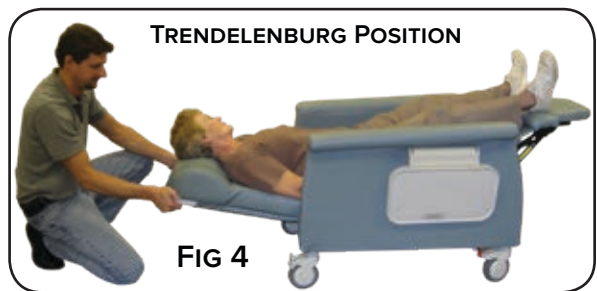

TRENDELENBURG POSITION

FIG 4

RETURNING CARE CLINER TO THE UPRIGHT POSITION



1. From the **RECLINED** position, grip the arm rests and simultaneously lift body weight upward while pulling on armrests. Continue until desired position is reached. **(FIG.1 - Above, Left)** Push down on footrest with feet to lower legrest. **(FIG.2 - Above Right)**
2. An attendant can also return the chair to the upright position by grasping the armrest with one hand, **AND** the top of the chair back with the other; then pull the back of the chair upright. **(FIG.3 - Below, Left)** The attendant can lower the footrest by pushing down on the footrest. **WARNING:** Keep hands and any other objects away from the mechanisms as injury may occur or cause damage to the chair. **(FIG.4 - Below, Right)**
3. **TRENDELENBURG POSITIONING MUST ONLY** be performed by an attendant. **(FIG. 3 & 4 - Below, Left & Right)** The gas spring **MUST** be engaged by squeezing the Release Handle at the top of the backrest. **(See page 6)**
 - **IMPORTANT:** Attendant must return the chair back to the full upright position after each use when patient exits the chair and before another patient enters chair.



LOCK/ UNLOCK CASTERS (FIG.1 - FIG.4)

* For 3" & 5" **Nylon** Casters and 5" **Steel** Casters

1. **Total-Lock casters** will lock wheel and swivel mechanism by using your foot & pushing down on the tab.

Location:

- Nylon Casters - All front and **RIGHT** rear (**LIGHT GREY TAB FIG.1 to FIG.2**)
- Steel Casters - All 4 casters (**FIG.3 to FIG.4**)

Directional-Lock casters aids with the steering of the chair.

NOTE: The directional-lock tab **ONLY** prevents the caster from swiveling. This caster can be installed on the **FRONT-LEFT** side of chair as a steering option.

Location:

- Nylon Casters - Left rear **ONLY** (**FIG. 1**)
- Steel Casters - **NOT AVAILABLE**





WARNING: LOCK CASTERS AT ALL TIMES, EXCEPT WHEN MOVING / TRANSPORTING CHAIR

Wheel-Lock casters (3" **BLACK** casters only) will lock wheel but does NOT prevent wheel from swiveling. Using your foot, push down on tab.

Location: All 4 **BLACK** casters installed on chair. (**FIG.5 to FIG.6 or FIG.7 to FIG.8**)

2. Release the locking mechanisms for ALL casters by pushing the above described tabs back to their upward position. (**NYLON FIG. 2 to FIG. 1**), (**STEEL FIG. 4 to FIG. 3**), (**3" BLACK FIG.6 to FIG.5 or FIG.8 to FIG.7**)

NOTE: All 3" **BLACK** casters are wheel locking **ONLY**. They **DO NOT** have a **TOTAL** and **DIRECTIONAL** lock feature.

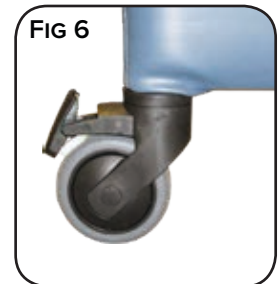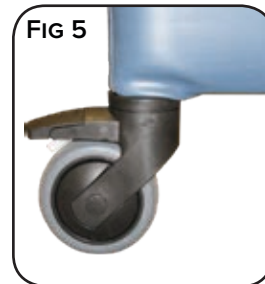

RAISE/ LOWER DROP-ARMS (FIG.1)

(For Drop-Arm Care Cliner Models **ONLY**)

BEFORE RAISING OR LOWERING, BE SURE USER IS CLEAR OF DROP-ARM.

1. **To Lower:** Grasp armrest in center; use free hand to **LIFT** the Drop-Arm Release Handle. Drop-Arm will disengage and you can then lower it to the fully dropped position. (FIG. 1)
2. **To Raise:** Firmly pull straight up in the center of the armrest. Make sure both armrests are locked into place before transporting user.

NOTE: Periodically lubricate the channels of the drop arms with your approved silicone base lubricant spray to keep the parts moving freely.



WARNING

DO NOT raise & lower Drop-Arms with tables in an upright position. (Drop-Arm Care Cliner Models **ONLY**)

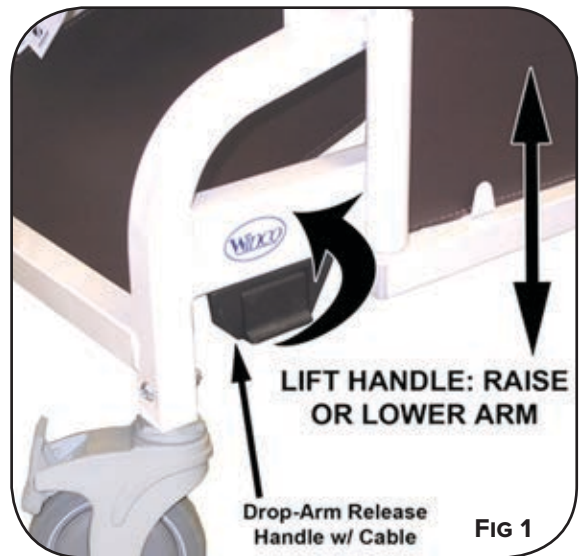


FIG 1

SIDE PANEL - REMOVAL / REPLACEMENT

(For Care Cliner Models **ONLY**)

1. For ease of cleaning and chair maintenance; Use a Phillips head screwdriver to unscrew two (2) screws at the front of chair and one (1) screw at the rear of chair, on inside of each panel. (FIG. 2)
2. Replace panel and re-install three (3) screws. (FIG. 2) when cleaning or maintenance of chair is complete.



FIG 2

DROP-ARM VINYL PANEL

(For Drop-Arm Care Cliner Models **ONLY**)

1. The vinyl panel is secured to the bottom of the Drop-Arm with a hook & loop fastener material. Peel hook & loop apart to allow ease of cleaning and access behind the Drop-Arm of the chair. (FIG. 3 & 4)
2. Re-attach by aligning and firmly pressing the hook & loop material back together.



FIG 3



FIG 4

SWING-ARM OPERATION

(For Models: 6740-6750-6940-6950 only)

1. To open, depress the red latch handle to unlock the swing-arm. **(FIG. 1)**
2. To close, push swing-arm toward latch until latch engages. **(FIG. 1)**



WARNING

6740 & 6750 Models: DO NOT have an attendee in the trendelenburg position (pg.6)

while both swing-arms are open. May cause tipping of chair.

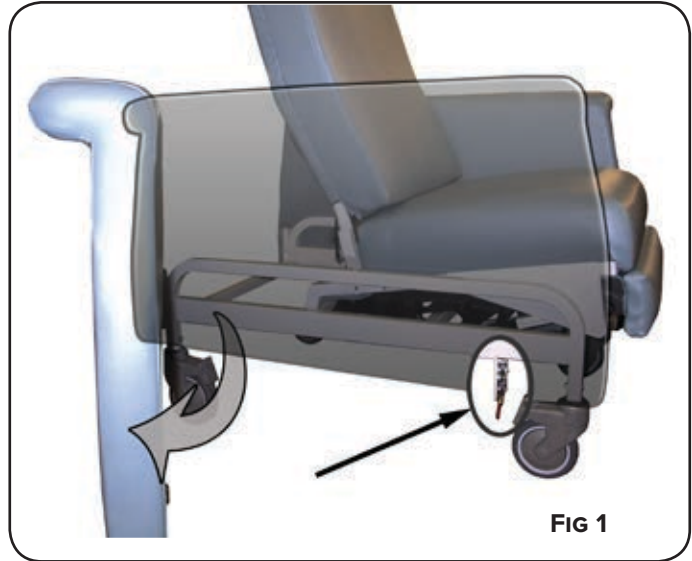


FIG 1

TO POSITION SIDE TABLES **(FIG 2)**

1. To raise table, grasp center of table and gently raise into place. Table will automatically lock into place when released.
2. To lower table, grasp table at the center, raise and pivot down in one movement.



FIG 2

PIVOT TABLE **(FIG 3)**

“Lap” Position

1. To place pivot-table in “lap” position, pivot the table away from chair and pull over lap.
2. Tighten star-knob to secure the pivot-table in the lap position.

“Exit” Position

1. With the table in the “lap” position, loosen star knob & push table away to exit chair.

Remove Pivot-Table

1. Loosen pivot-table star knob
2. Pull up on pivot-table until the table stem comes out of the black bracket.

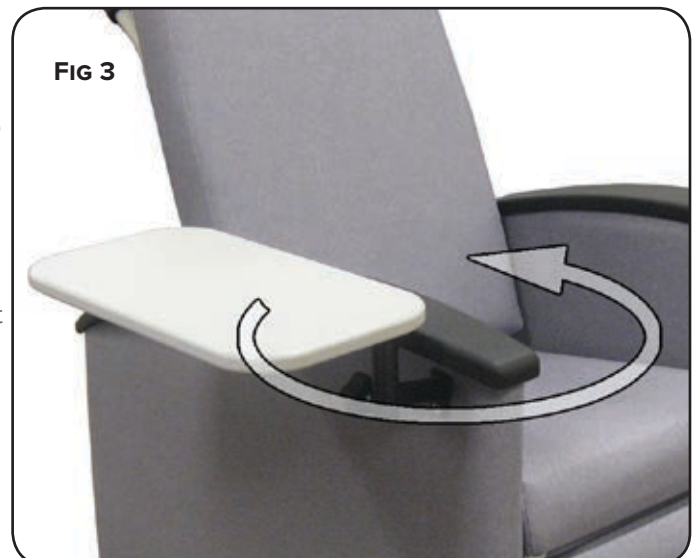


FIG 3



WARNING

DO NOT lean or sit on side tables. Tables may break and cause injury. **DO NOT** allow



patients to raise from chair using side tables.

GENERAL MAINTENANCE

ADJUSTING THE GAS SPRING



NOTE: Tension or "ease of movement" of the gas spring is set by factory and cannot be changed. The following instruction is for adjusting the gas spring "release and engagement points" for Trendelenburg or "deep recline" positions only.

1. Lock Casters (page 8). Remove cable from Operator and loosen 11/16" hex nut on top of Operator. **(FIG 1)**
2. Have a second person hold the back of chair steady and remove the Bow-Tie Cotter Pin and Clevis Pin. **(Fig 2)** The Operator can now be rotated on threaded end of Gas Spring.
3. If Back of chair does not go into Trendelenburg by squeezing handle; Rotate Operator as shown, one (1) full revolution. **(FIG 3A)** If Back of chair goes into Trendelenburg **WITHOUT** squeezing handle; Rotate Operator as shown one (1) full revolution. **(FIG 3B)**
4. Once you make an adjustment; Replace Clevis Pin and Bow-Tie Cotter Pin. Squeeze release handle and recline the back, to check for proper adjustment. Repeat above steps as needed until correct release action is achieved. **TIGHTEN HEX NUT** (that was loosened in step 1) once Operator is adjusted properly.

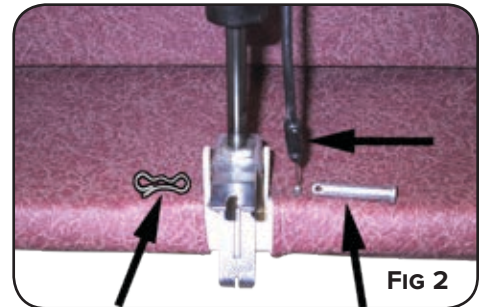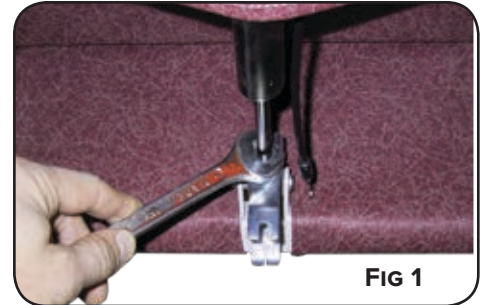




ADJUSTING THE RECLINER MECHANISMS

1. Place chair on its side in the 2nd recline position. Each chair has a L/R recline mechanism with two (2) tension adjustments on each - Four (4) total. Tension adjustment **A** controls the footrest and legrest action. Tension adjustment **B** controls the recline action from the 2nd to 3rd recline positions. **(FIG 4)**
2. Locate the two hex nuts (**A or B**) on the L&R recline mechanisms. Use a 7/16" wrench and adjust both L&R hex nuts one full revolution. R to tighten or L to loosen. **(FIG 5)**
3. Return recliner to upright position and check recliner for reclining or legrest / footrest action. Repeat step two if needed. Tighten hex nuts no more than two full revolutions.

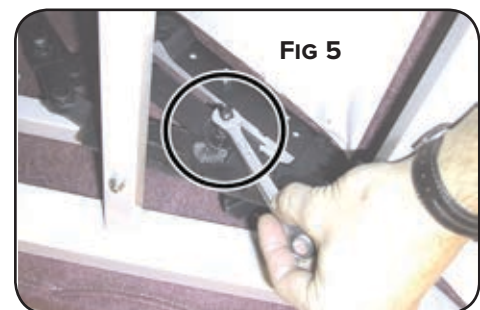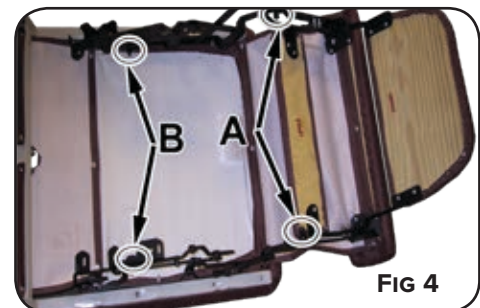

REPLACING THE RECLINING MECHANISM

1. Remove Care Cliner back and put aside. (see Back Assembly Instructions page)
2. Position recliner so that the recline mechanism, located on the chair frame, is accessible. Remove the 4 bolts that attach the recline mechanism to the chair frame. **(FIG 1)**
3. Remove the recline mechanism, and the attached Seat Frame assembly from the chair Main Frame. Turn the Seat Frame & mechanism upside down on a table or other stable surface. **(FIG 2)**
4. Remove all screws that attach Footrest, Mid-Ottoman, Legrest, and Seat from the damaged recline mechanism. **(FIG 3)**
5. Re-attach the Seat, Legrest, Mid-Ottoman, and Footrest to the new mechanism. Re-assemble the Seat/Recline Mechanism Assembly to the chair frame using (4) new locknuts and the bolts that were previously removed. **(FIG 1)**

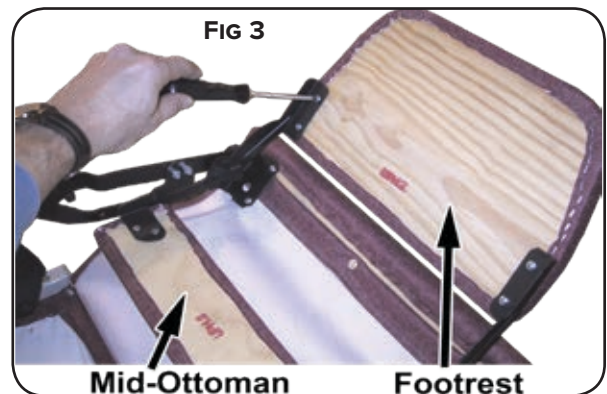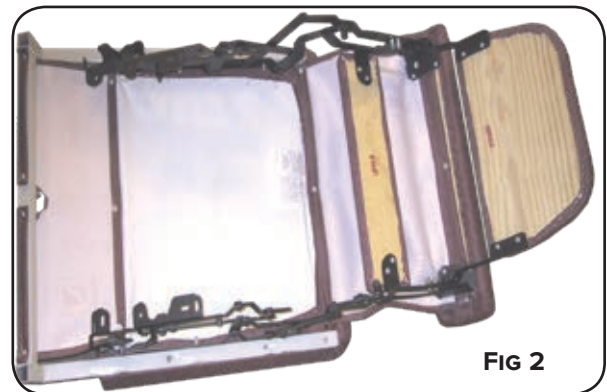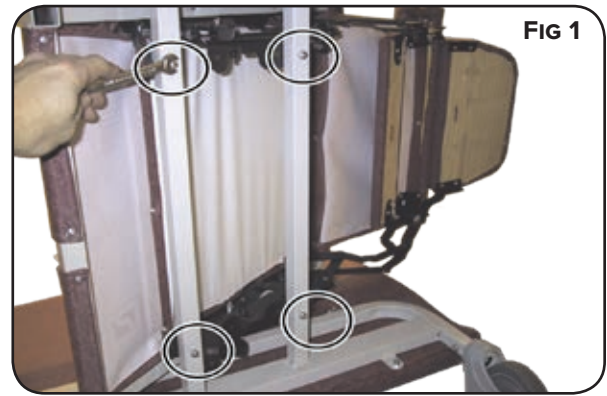

IMPORTANT!

REGULARLY lubricate pivot points on the Recline Mechanism with a silicone based lubricant. Once per week, position the Chair Back into full recline position, squeeze release handle and go to full Trendelenburg position. This will keep the gas cylinder lubricated. Periodically recheck tightness of all screws, bolts and nuts.

GENERAL CARE AND CLEANING

Remove hair & body oils; Your recliner should be cleaned with mild soap & water solution and a damp cloth on a regular basis (especially where skin & hair make contact with upholstery). Avoid harsh detergents that could damage the upholstery or finish of your recliner. If the recliner is disinfected, the chair **MUST** be wiped off with **ONLY** clean water on a damp cloth.

Note: Disinfectants alone, will not provide adequate "grease or oil cutting" properties to remove hair & skin oils. Over time, the appearance and feel of your upholstery may diminish if not cleaned properly. When cleaning or disinfecting the inside of a swing arm, it is important to allow the arm to air dry before closing the arm or wipe with clean dry cloth before closing. Otherwise, the moisture between the arm and the frame could cause the frame to rust.

PARTS LIST: For information on parts for these recliners, please visit wincomfg.com

SPECIFICATIONS

6530-6531 CARE CLINER



SHOWN: 6530

*See Chair for Maximum Weight Capacity

Seat Height	21"	Overall Width	32.25"
Seat Width	21"	Back Height Above Seat	31.5"
Back Width	21"	Seat Depth	20"
Overall Height	49"	Weight	94 lbs.

6540-6541 XL CARE CLINER



SHOWN: 6540

*See Chair for Maximum Weight Capacity

Seat Height	21"	Overall Width	36.25"
Seat Width	25"	Back Height Above Seat	31.5"
Back Width	25"	Seat Depth	20"
Overall Height	49"	Weight	111 lbs.

6700-6704, 6710-6714 PREMIER CARE CLINER



SHOWN: 6700

*See Chair for Maximum Weight Capacity

Seat Height	20.5"	Overall Width ... 670 = 32" 671 = 36"
Seat Width ... 670 = 22" 671 = 26"		Back Height Above Seat 20.5"
Back Width ... 670 = 22" 671 = 26"		Seat Depth
Overall Height	47"	Weight ... 670 = 95 lbs. 671 = 122 lbs.

6740, 6750 SWING-ARM PREMIER CARE CLINER



SHOWN: 6740

*See Chair for Maximum Weight Capacity

Seat Height	21"	Back Height Above Seat ... 674 = 30"
Seat Width	674 = 24" 675 = 28"	675 = 32"
Overall Height	46.5"	Seat Depth
Overall Width ... 674 = 34" 675 = 38"		Weight ... 674 = 125 lbs. 675 = 135 lbs.

NOTE: Gas Spring

6530-1/6550-1/670#/6740/6900/6940 = 1 Gas Spring / 6540-1/6570-1/671#/6750/6910/6950 (XL) = 2 Gas Springs.

SPECIFICATIONS

6550-6551 DROP-ARM CARE CLINER



SHOWN: 6550

*See Chair for Maximum Weight Capacity

Seat Height	21"	Back Height Above Seat	31.5"
Seat Width	21"	Seat Depth	20"
Back Width	21"	Drop-Arm Height to Floor	21"
Overall Height	49"	Weight	105 lbs.
Overall Width	33.75"		

6570-6571 XL DROP-ARM CARE CLINER



SHOWN: 6570

*See Chair for Maximum Weight Capacity

Seat Height	21"	Back Height Above Seat	31.5"
Seat Width	25"	Seat Depth	20"
Back Width	25"	Drop-Arm Height to Floor	21"
Overall Height	49"	Weight	111 lbs.
Overall Width	37.75"		

6900-6910 ELITE CARE CLINER



SHOWN: 6910

*See Chair for Maximum Weight Capacity

Seat Height	21"	Overall Width .. 690 = 33.5" 691 = 37.5"
Seat Width ... 690 = 21" 691 = 25"		Back Height Above Seat 32.5"
Back Width ... 690 = 21" 691 = 25"		Seat Depth
Overall Height	48.5"	Weight ... 690 = 100 lbs. 691 = 123 lbs.

6940-6950 SWING-ARM CARE CLINER



SHOWN: 6940

*See Chair for Maximum Weight Capacity

Seat Height	21"	Overall Width	694 = 33.5"
Seat Width	694 = 21" 695 = 25"		695 = 37.5"
Back Width ... 694 = 21" 695 = 25"		Back Height Above Seat	32.5"
Overall Height	48.5"	Seat Depth	21.5"
		Weight... 694=104lbs. 695=127lbs.	

NOTE: ALWAYS refer to the weight capacity label on your chair. In all cases the labeling on the chair (at the time of delivery) is the correct rating for your chair - that weight should **NOT** be exceeded.

HEAT & MASSAGE OPERATING INSTRUCTIONS

HEAT & MASSAGE OPTIONS

Control is located on the inside of right arm of chair.



HEATING: Gently touch **HEAT** button - See indicator light on switch

- 3 Lights** = High Heat
- 2 Lights** = Medium Heat
- 1 Lights** = Low Heat
- No Light** = HEAT OFF

Allow 5-8 minutes for back and seat to warm to desired **TEMPERATURE** setting.

Heat will automatically shut off after approximately 60 minutes. (Timer resets when switch is pressed)

MASSAGE (OPTIONAL): Gently touch **MASSAGE** button

- 1X** = Lower Back / Low Intensity
- 2X** = Lower Back / High Intensity
- 3X** = Upper & Lower Back / Low Intensity
- 4X** = Upper & Lower Back / High Intensity
- 5X** = Alternating Upper & Lower Back
- 6X** = MASSAGE OFF

Massage will automatically shut off after approximately 15 minutes. (Timer resets when switch is pressed)

CARE & TROUBLE-SHOOTING TIPS

- Use a **DAMP CLOTH ONLY** to clean switch, wiring or power pack (located under seat frame).
- If there is no heat or massage or flashing light on switch - check power cord connections to wall outlet **AND** connection of power cord to power pack unit under chair.
- If switch displays a flashing light; Check all accessible wire connectors under chair for possible loose connections, then contact Winco customer service: 1-800-237-3377.



Rear View of Chair FIG 1

CORD STORAGE (Models with Heat or Massage only)

Before transporting chair, or when heat & massage is not in use, power cord should be coiled and stored in the "Hook & Loop" fastener provided as shown, to prevent tripping on cord, damage to cord, damage to recliner, etc... **(FIG 1)**

CARE CLINER OPTIONS

HEAT / HEAT & MASSAGE

HM Heat & Massage

HT Heat ONLY

AVAILABLE FOR ALL MODELS



IV POLE & ATTACHMENT

IV IV Pole & Attachment

AVAILABLE FOR ALL MODELS



LEG EXTENSIONS

RE00 Round Leg Extension
FITS: 653 & 654 MODELS

SE00 Square Leg Extension
FITS: 655, 657, 690, 691, 694 & 695 MODELS



PIVOT TABLE

AVAILABLE FOR: 670, 671, 674, 675,
690, 691, 694 & 695 MODELS

For LEFT or RIGHT Side



OXYGEN TANK HOLDER

O2 Oxygen Tank Holder

AVAILABLE FOR: 670, 671, 674, 675,
690, 691, 694 & 695



SIDE CUSHION

SC00 Side Cushion

AVAILABLE FOR ALL MODELS



ARM REST COVERS

AC00 Arm Rest Covers (Pair)

AVAILABLE FOR: 690, 691, 694 &
695 MODELS



ACCESSORY HOOK

AVAILABLE FOR: 670, 671, 674, 675,
690, 691, 694 & 695 MODELS

For LEFT or RIGHT Side



WARRANTY INFORMATION

WINCO MEDICAL FURNISHINGS LIMITED WARRANTY

TERMS

Winco Mfg., LLC., warrants Winco Medical Furnishing products to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

- **Limited One Year Complete Product Warranty:** Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC's sole discretion, it may repair or replace any components that are found to be defective during the first year after shipment from the Winco facilities. Winco Mfg., LLC, shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.
- **Limited Three-Year Warranty:** Winco Mfg., LLC warrants the recliner mechanisms, casters, electrical components, vinyl [from cracking or delaminating], and pressurized gas springs, for a period of three (3) years. At Winco Mfg., LLC's sole discretion, it may repair or replace components found to be defective. Winco Mfg., LLC, shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.
- **Limited Lifetime Chair Frame Warranty:** Winco Mfg., LLC warrants the steel product frames, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or like legislation or statute are hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow exclusions or limitations so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights. In addition, you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit
2. Accident, alteration, abuse or misuse of product i.e., exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions
3. Fire, water damage, theft, war, riot, hostility, acts of God

Examples contained in this list are not to be construed as all-inclusive.

Rev. 06/01/2018 TL



CONTACT US

To arrange a demonstration or to speak with one of our associates, please feel free to contact us at:
800-237-3377 | WincoMfg.com | 5516 SW 1st Lane, Ocala, FL 34474 | Info@WincoMfg.com